



3M™ Fit Test Apparatus FT-10, FT-20 and FT-30 – Introduction

Welcome and thank you for choosing a 3M™ Qualitative Fit Test Apparatus.

During this short review, we will be discussing how to use your Fit Test Apparatus. The goal of this review is to show you how to set up and operate your Fit Test Apparatus.

Additional detailed information can be found in your User Instructions packaged with the product or by calling the 3M Technical Service Hotline at 1-800-243-4630




Double click picture to start video

Click button to view video text

- ◆ This document has been encoded for a Windows operating system
- ◆ This version of the interactive document requires at minimum an internet connection with (256Kbs) minimum connection speed and Acrobat Reader 6.0 or higher for proper operation
- ◆ If there is no video, reaffirm that you are running Acrobat Reader 6.0 (minimum requirement)
- ◆ If video problems persist, try closing other applications that may be consuming the required memory
- ◆ If there is no audio or the volume is too low, right-click the audio setting icon and adjust accordingly
- ◆ Sound but no video, click NEXT



- ◆ Still having problems, call 1-800-243-4630

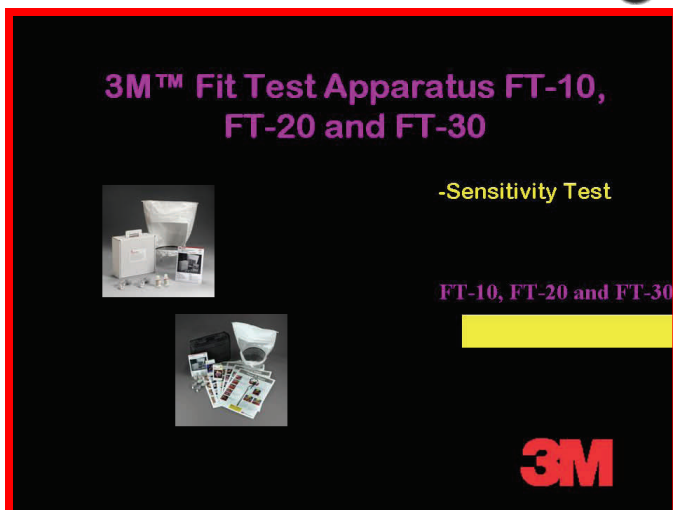

Double click picture to start video

Click button to view video text



3M™ Fit Test Apparatus FT-10, FT-20 and FT-30



-Fit Test

FT-10, FT-20 and FT-30



3M

3M™ Qualitative Fit Test Apparatus FT-10, FT-20 and FT-30

Fit Test Apparatus Replacement Parts

Part No.:	Description:	Replace When:
FT-11	Sensitivity Solution - Sweet	Empty
FT-12	Fit Test Solution - Sweet	Empty
FT-31	Sensitivity Solution - Bitter	Empty
FT-32	Fit Test Solution - Bitter	Empty
FT-13	Nebulizer	Damaged
FT-14	Test Hood	Ripped, torn or damaged
FT-15	Collar	Damaged

**WARNING**

Use of these products by untrained or unqualified persons, or use not in accordance with the instructions, may adversely affect product performance and **result in sickness or death**. These products are to be used only by qualified persons who are properly trained in their use and maintenance, and only in accordance with their operating and maintenance manuals. Each person using these products must first read and understand the operators manual.

For proper use, see your supervisor, product instructions and warnings or call 3M OH &ESD Technical Service at 1-800-243-4630. In Canada call 1-800-265-1840 extension # 613

For more information, please contact:**3M Occupational
Health and Environmental Safety Division
(OH&ESD)****In the U.S., contact:****Sales Assistance**

1-800-896-4223

Technical Assistance

1-800-243-4630

Fax On Demand

1-800-646-1655

Internet<http://www.3M.com/occsafety>**For other 3M products**

1-800-3M HELPS

In Canada, contact:

3M Canada Company, OH&ESD

P.O. Box 5757 London, Ontario N6A 4T1

Sales Assistance

1-800-265-1840, ext. 6137

Technical Assistance (Canada only)

1-800-267-4414

Fax On Demand

1-800-646-1655

Internet<http://www.3M.com/CA/occSafety>**Technical Assistance In Mexico**

1-800-712-0646 5270-2255, 5270-2119 (Mexico City only)

Technical Assistance In Brazil

0800-132333

Fax On Demand O.U.S. Locations

1-651-732-6530

3M Occupational Health and Environmental Safety Division
3M Center, Building 235-2W-70
St. Paul, MN 55144-1000

©3M September 2006. All rights reserved