

Ortholux[™] Luminous Curing Light Technique Guide

Installation




1. Read instructions



2. Plug charger into wall outlet



3. Attach the light guide

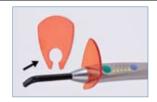

4. Insert the eye-shield



5. Remove battery cap and insert battery into handpiece. Tighten by turning clockwise




6. Charge unit for two hours

Operation



1. If handpiece is in SLEEP mode, briefly press START button to wake up unit

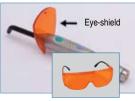

Use eye-shield and/or orange glasses




3. Press TIME button to select exposure time



4. Briefly press and release START button to activate selected exposure time



5. For Tack Cure, press and hold START button

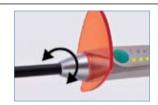


 Rotate light guide independent of eye-shield to achieve desired light guide position and eye protection during cure



7. Hold light guide 1-2 mm above appliance, perpendicular to tooth surface

Maintenance

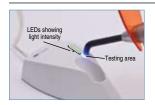

 Measure light intensity by placing light guide tip flat on testing area located on charger base and activating the light



2. Remove and inspect both ends of light guide for cracks, chips, dried adhesive, etc.



3. Clean both ends of light guide with a soft cloth. Light guide can be autoclaved



 Clean charger, handpiece, eye-shield by spraying disinfectant on a towel and wiping down



5. Wipe dry all parts before use



6. To remove battery from handpiece turn counter-clockwise



3M Unitek Orthodontic Products 2724 South Peck Road Monrovia, CA 91016 USA www.3MUnitek.com In U.S. and Puerto Rico: 1-800-423-4588 • 626-574-4000 In Canada: 1-800-443-1661 Technical Helpline: 1-800-265-1943 • 626-574-4577 CE Hotline: 1-800-852-1990 x4649 • 626-574-4649 Outside these areas, contact your local representative.

Please recycle. Printed in USA. © 2009 3M. All rights reserved. 012-198 0904



OrtholuxTM Luminous Curing Light



