



3M™ Bair Hugger™ therapy forced-air warming

Keep your Patients Warm



Lead
The Way

3M

Anesthetized patients cannot regulate their temperature.

The induction of anesthesia can cause a rapid drop in core body temperature and increase the risk of hypothermia. The adverse and costly outcomes associated with unintended hypothermia are well documented.* By providing warmth with 3M™ Bair Hugger™ forced-air and fluid warming products, you can effectively prevent hypothermia in your patients.

**Studies available upon request. Most studies performed on human patients.*

Forced-Air Warming & Fluid Warming



Bair Hugger Warming Unit

Model 50577 New

Model 59577 Refurbished



Bair Hugger 241 Fluid Warming Set

Model 24100

3M™ Bair Hugger™ Warming Units

Bair Hugger warming units are the standard of care for forced-air warming therapy.

- Quiet and easy to use
- Small size provides more usable work space
- Freestanding, IV pole, bedrail and rolling cart mounting options available
- Built-in hour meter makes it easy to monitor usage for preventative maintenance
- Easy filter changes

3M™ Bair Hugger™ 241™ Fluid Warming Set

The Bair Hugger 241 fluid warming set enables you to use two warming modalities with one piece of equipment – the Bair Hugger warming unit. Just slide the 241 set into a Bair Hugger warming unit hose and you instantly add fluid warming.

- No stand-alone fluid warmer required
- In-line temperature display on outlet tubing monitors temperature
- Infuse up to three liters per hour (KVO to 3,000 mL/hr)

Bair Hugger therapy provides safe, effective warming for your patients.

Bair Hugger Animal Health Blankets

Bair Hugger blankets feature unique, flexible designs that provide safe and effective warming for your patients. A variety of designs are available to help you warm patients of all shapes and sizes. With Bair Hugger therapy you don't have to worry about leaky water mattresses caused by claw punctures.

More than 165 million human patients have experienced the benefits of Bair Hugger forced-air warming blankets and 3M™ Ranger™ blood/fluid warming technology. Now your animal patients can, too.



Underbody Blanket
Model 55577




Small Blanket
Model 53777




Long Blanket
Model 53077

3M™ Bair Hugger™ Therapy

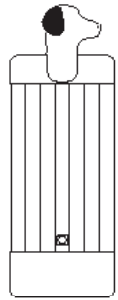


Large Blanket

Model 30077
84 x 36 in

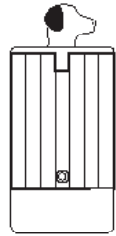


Medium Blanket

Model 31077
60 x 36 in

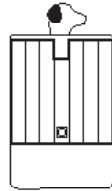


Small Blanket

Model 53777
35 x 24 in

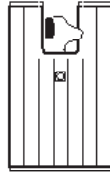

Draped Blanket

Model 54077
42 x 36 in
Drapes: 24 x 24 in

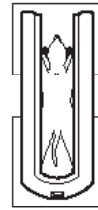


Long Blanket

Model 53077
74 x 18.5 in
Drapes (2): 24 x 18 in




Underbody Blanket

Model 55577
36 x 33 in
Drapes (2): 24 x 24 in

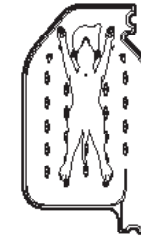

Large Underbody Blanket

Model 55077
60 x 32 in
Drapes (2): 24 x 24 in

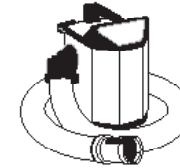

Warming Unit


Model 60577 - New
Model 60677 - Refurbished
13h x 10w x 11d in
13.8 lb



Warming Unit Filter


Model 90000
(for Models 60577 & 60677)
Change filter every 12 months or after
500 hours of use to ensure optimal performance

3M™ Ranger™ Fluid Warming System

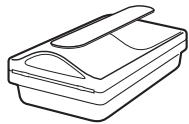

Warming Unit

Model 24577 - New
Model 29577 - Refurbished
4.5h x 7.5w x 10d in
7.7 lb

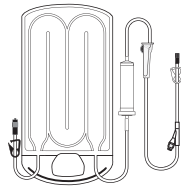

Standard Flow Set

Model 242
Flow Rates: KVO to 9,000 mL/hr
Priming Volume: 39 mL
Pressure Infusion: Up to 300 mm Hg



241 Blood/Fluid Warming Set

Model 241
Flow Rates: KVO to 3,000 mL/hr
Priming Volume: 34 mL

For more information, please contact your 3M representative or call Customer Service at 1-800-228-3957.

www.keepanimalswarm.com



Infection Prevention Division
3M Health Care
3M Center, Building 275-4E-01
St. Paul, MN 55144-1000
U.S.A.
800-228-3957
3m.com/infectionprevention

3M is a trademark of 3M Company, used under license in Canada. BAIR HUGGER, RANGER, 241 and the BAIR HUGGER logo are trademarks of Arizant Healthcare Inc., used under license in Canada. ©2013 Arizant Healthcare Inc. All rights reserved. www.keepanimalswarm.com
601668E 11/13



Ranger™
Blood and Fluid Warming Systems