

APC™ II


Adhesive Coated Appliance System

APC™ II APC™ II APC™ II APC™ II APC™ II

Faster, more efficient bonding than any other system.

In 1991, 3M Unitek set the standard in orthodontic bonding efficiency with the introduction of APC™ Adhesive Coated Brackets.

Today, the innovative APC™ II Adhesive Coated Appliance System takes bonding ease and efficiency to the next level. A soft, workable APC II adhesive. Dramatically easier etching/priming with Transbond™ Plus Self Etching Primer and faster curing with the Ortholux™ LED Curing Light.

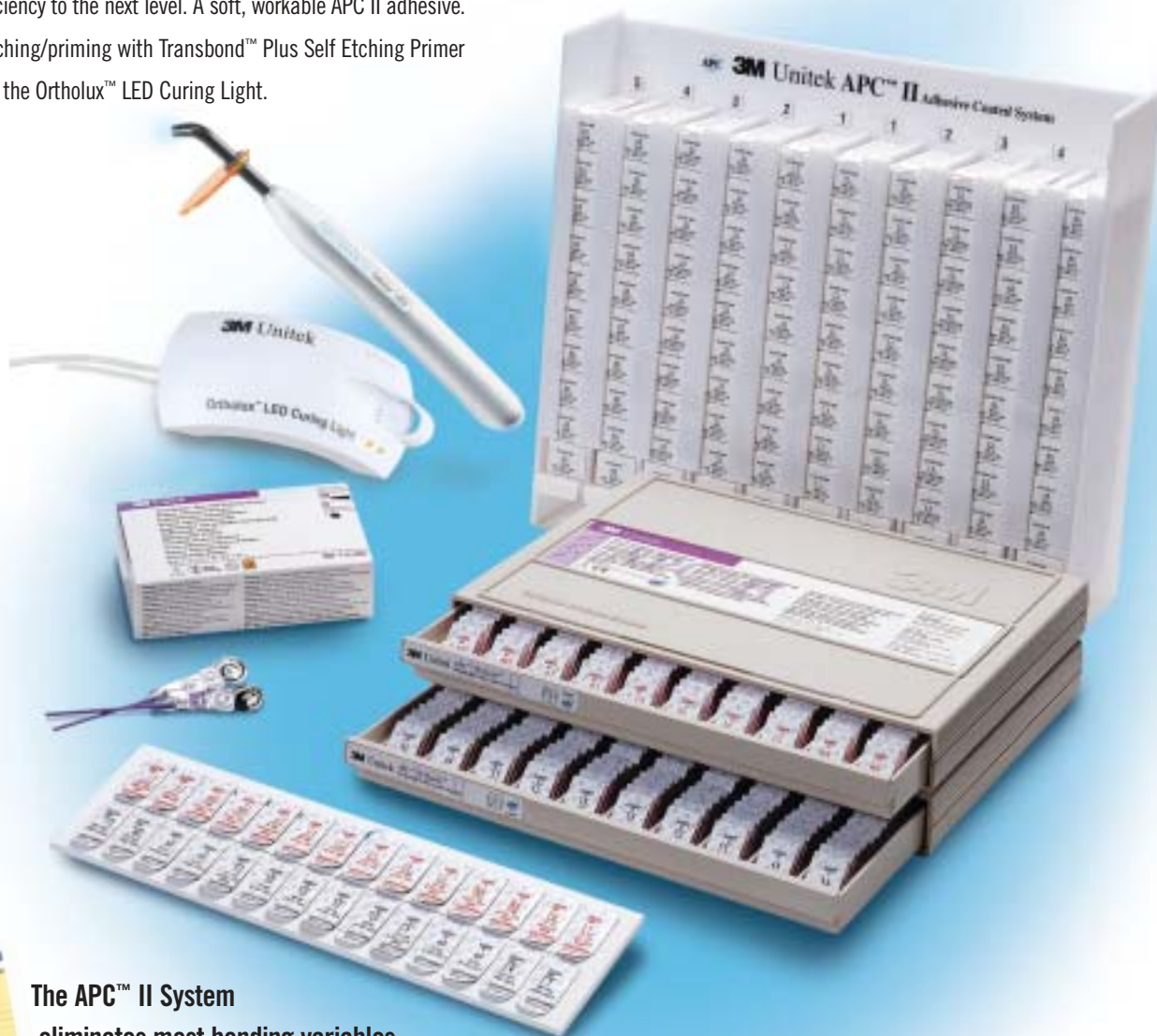

The APC™ II System eliminates most bonding variables.

The APC II System precoats each bracket with adhesive. There's no mixing, no application, no opportunity to contaminate the base. Add Transbond Plus Self Etching Primer that performs under both wet and dry conditions, and you've virtually eliminated bonding variables.




3M Unitek

Products that make your life easier.

Fewer steps, less time with the APC™ II System.



Traditional Light Cure Bonding = 11 steps.

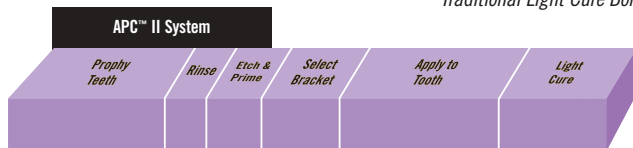

APC™ II System = 6 steps.

Faster Curing.

The APC II System cuts curing time and gives you portable cordless convenience when the Ortholux™ LED Curing Light is used. Faster cure, less total bonding-process time.



Less Waste and Clean-up.

The APC II System gives your staff a break every day. No messy mixing. No bracket handling problems with each individual bracket clearly identified and pre-oriented on a special foam liner that virtually eliminates bracket rotation in transit. No wasted time getting to chair side.

The APC II System with Transbond™ Plus Self Etching Primer takes you from prophylaxis to cure in fewer steps than any other bonding system available today. No separate etching and priming. No drying. No adhesive mixing. No adhesive application.

APC™ II Adhesive Coated Appliance System. The most efficient bonding system in orthodontics.

Advanced Formula Adhesive

APC II adhesive viscosity has been formulated to deliver a soft, pliable feel for easier positioning. Bracket drift is virtually eliminated. Minimal adhesive flash eases clean up.

Self Etching Primer

Transbond Plus Self Etching Primer etches and primes in one simple step without any compromise in bond strength. Proven performance in both wet and dry conditions.



Cordless Curing Light

The Ortholux LED Curing Light offers curing times that are fast and you get cordless portability.



Dispensing System

Each APC II adhesive coated bracket comes in an individual color-coded well for easy identification. A tamper-evident heat seal ensures a 3-year shelf-life. A compact dispensing system allows storage of up to 20 cases of brackets by slot size, tooth and prescription. A convenient patient set up tray allows you to transfer exactly what is needed to chairside.



3M Unitek
Orthodontic Products
2724 South Peck Road
Monrovia, CA 91016 USA
www.3MUnitek.com

In U.S. and Puerto Rico: 1-800-423-4588 • 626-574-4000
In Canada: 1-800-443-1661
Technical Helpline: 1-800-265-1943 • 626-574-4577
CE Hotline: 1-800-852-1990 x 4649 • 626-574-4649
Outside these areas, contact your local representative.



Printed on 50% recycled waste paper, including 15% post-consumer waste paper.

© 2005 3M Company.
All rights reserved.
REF 016-937 0503