

Unitek™ Saliva Ejector/ Tongue Holder


Instructions For Use



Assembly Instructions

Perforations in the plastic tubing must face up (occlusally). See Fig. 1.

Fig. 1

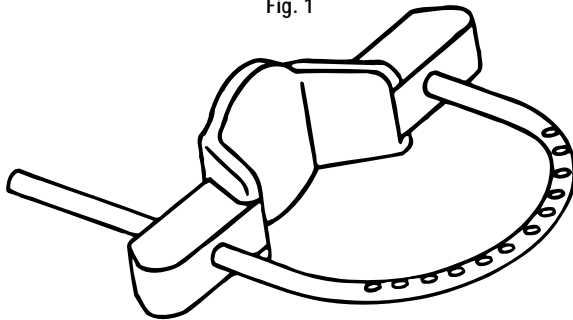

If saliva ejector assembly is distorted or the perforations in the tubing are disoriented, correction can be made as follows:

1. Place assembly in hot water (180°F) for approximately 20 seconds.
2. Twist plastic tubing to reorient holes as shown in Fig. 1.
3. Cool in cold water for 30 seconds.


Instructions For Use

Important: This item is for single patient use only. Discard after use to avoid possible patient cross contamination.

Place moisture control devices before placing Saliva Ejector/Tongue Holder.

1. Insert the Saliva Ejector/Tongue Holder into the patient's mouth, folding the perforated tubing loop under the tongue. Rest the mouthguard blocks on the mandibular first molars. (The mouthpiece can be trimmed with scissors, and smoothed with pumice and a rag wheel for a more comfortable fit). See Fig. 2.

Fig. 2

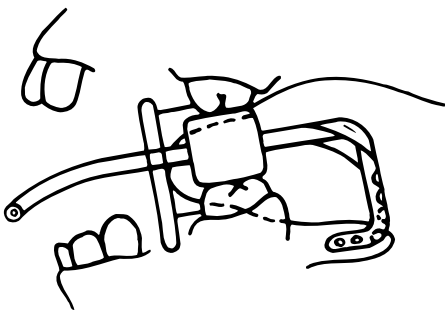

2. Connect the extraoral length of tubing to the adapter, and connect the adapter to a vacuum source. (The rubber fitting may be required to make this connection).
3. Turn on the vacuum source, and verify that the Saliva Ejector/Tongue Holder is operating.

After Use

1. Remove the Saliva Ejector/Tongue Holder from the mouth.
2. Disconnect the adapter from the vacuum source.
3. Disconnect the adapter from the Saliva Ejector/Tongue Holder.
4. Immerse the adapter in disinfectant solution, and retain for reuse.

Order

REF Number	Description
REF 709-020	Saliva Ejector/Tongue Holder

Caution: U.S. Federal Law restricts this device to sale by or on the order of an orthodontic professional.

3M

**3M Unitek
Orthodontic Products**
2724 South Peck Road
Monrovia, CA 91016 USA

© 2004 3M Company.
All rights reserved.
REF 011-388-5 0509

CE
0086 93/42/EEC

European Representative:
3M Unitek GmbH
ESPE Platz
82229 Seefeld, Germany
0049/(0)8152-397-0