

# Particulate Respirator 8612F and 8670F For Use by the General Public in Public Health Medical Emergencies

Issue Date 4/11/08

## Helping the General Public Prepare for Medical Emergencies

3M particulate respirators cleared by the U.S. Food and Drug Administration (FDA) are for use by the general public in public health medical emergencies, such as an influenza pandemic, to help reduce wearer exposures to airborne germs. They are not intended for any other use.

## Discover the Benefits of the 8612F and 8670F:


- NIOSH N95 certified filtration efficiency
- Lightweight, off-the-face, collapse-resistant designs help provide improved comfort and breathability
- Single size designed for adult face sizes and shapes (not for use by children)
- Single-use, disposable formats – discard in closed waste container after wearing

## The 8612F also features:

- Braided headbands for comfort against face and hair
- Pre-molded, cup style structure that conforms to provide a snug fit

## And the 8670F offers:

- Unique three-panel flat fold design for convenient portability
- Individually packaged to help ensure product cleanliness during storage and transport

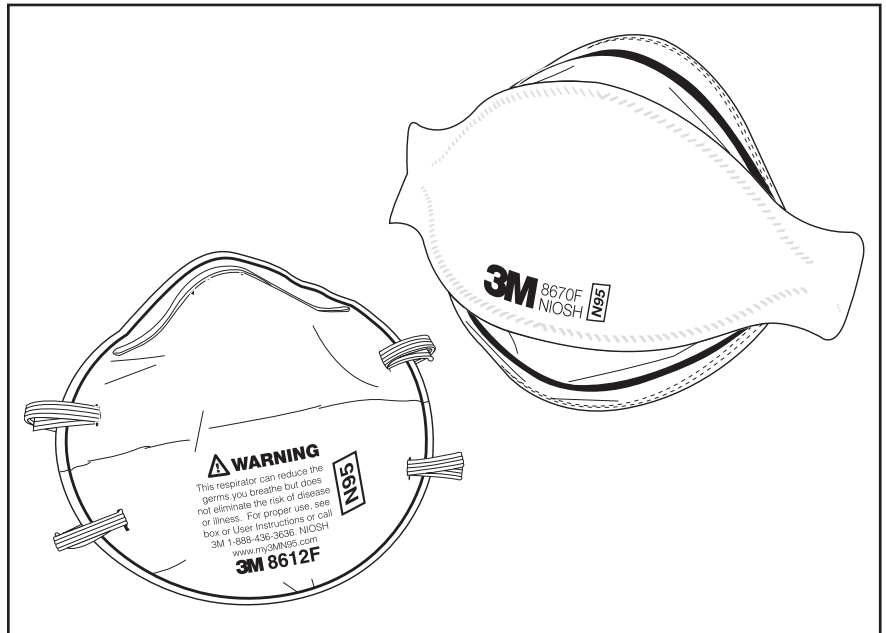

3M™ Particulate Respirator 8612F and 8670F

## How do these FDA-cleared respirators differ from other N95 respirators?

The Food and Drug Administration (FDA) requires these respirators for use by the general public in public health medical emergencies to have an N95 rating, which means the products meet NIOSH 95 certification requirements. Unlike other N95 respirators currently on the market, 3M's 8612F and 8670F have been cleared by the FDA for use by the general public in a public health medical emergency. The package labeling and *User Instructions* have been specifically designed to communicate the information needed by consumers to understand the role of respirators in a medical emergency, as well as how to use the respirator properly. 3M is also providing additional training materials developed for the general public on our website [www.my3MN95.com](http://www.my3MN95.com).

**Important Safety Instructions**

For your respirator to help reduce the number of germs you breathe, you must read and follow the *User Instructions* provided with the product packaging.

**IMPORTANT**

If you have a pre-existing lung disease such as asthma or emphysema, underlying heart disease such as heart failure or other health conditions, you may have difficulty breathing through respirators and should consult your healthcare provider (doctor) before use.

**Select the Right Respirator**

- The U.S. FDA has cleared respirator models 8612F and 8670F for use by the general public in public health medical emergencies, such as an influenza pandemic, to help reduce wearer exposures to airborne germs.
- Not intended for any other use.
- NIOSH N95 certified for filtration efficiency.

- Not for occupational use.
- These are adult size respirators and are not for use by children.
- These respirators are fluid resistant to splash and spray of body fluids.

**Wear it Right**

- The respirator must be worn whenever you are exposed to germs during a public health medical emergency, such as an influenza pandemic.
- Always follow the User Instructions inside the box when you wear the respirator.
- Practice putting the respirator on to make sure you are ready in the event of a public health medical emergency.
- Anything that comes between the respirator and your face will make the respirator less effective by interfering with its fit.

**Check Your Fit**

- Check your fit to ensure you are getting a good seal.


- Putting the respirator on correctly means more of the air you breathe goes through the respirator filter.
- Always follow instructions in Section 6 of *User Instructions*, How To Wear, to check your fit.

**Use it Correctly**

Although these respirators can help reduce the number of germs you breathe, they will not eliminate the risk of disease or illness. See *User Instructions* and Center for Disease Control (CDC) website, [www.cdc.gov](http://www.cdc.gov), for information on other things you can do to help reduce your risk of getting sick.

**Ordering Information**

Product #	UPC	SKU	Respirators/ Box	Boxes/ Case	Each/ Case
Particulate Respirator 8612F	5-00-51135-89167-9	70-0714-8609-0	20	6	120
Particulate Respirator 8670F	5-00-51135-89168-6	70-0714-8610-7	20	12	240

<b>AWARNING</b>	
	<p>This respirator is intended to help reduce breathing in pathogenic biological airborne particulates or germs during public health medical emergencies, such as an influenza pandemic. If you do not follow all instructions and limitations on the use of this respirator and/or do not wear this respirator during all times of exposure to the airborne germs the respirator may not be as effective. The respirator cannot stop the breathing in of all germs in the air and does not eliminate the risk of disease or illness. For proper use of this respirator, see the box and <i>User Instructions</i> or call 3M in the U.S.A. at 1-888-436-3636.</p>

**3M Occupational Health and Environmental Safety Division**  
3M Center, Building 235-2W-70  
St. Paul, MN 55144-1000

<p><b>For more information, please contact:</b></p> <p><b>3M Occupational Health and Environmental Safety Division (OH&amp;ESD)</b></p> <p><b>In the U.S., contact:</b></p> <p><b>Sales Assistance</b> 1-800-896-4223</p> <p><b>Information Line</b> 1-888-436-3636</p> <p><b>Technical Assistance</b> 1-800-243-4630</p> <p><b>Fax On Demand</b> 1-800-646-1655</p> <p><b>Website</b> <a href="http://www.my3MN95.com">www.my3MN95.com</a> <a href="http://www.3M.com/OccSafety">www.3M.com/OccSafety</a></p> <p><b>For other 3M products</b> 1-800-3M HELPS</p> <p><b>In Canada, contact:</b> 3M Canada Company, OH&amp;ESD P.O. Box 5757 London, Ontario N6A 4T1</p> <p><b>Sales Assistance</b> 1-800-265-1840, ext. 6137</p> <p><b>Technical Assistance (Canada only)</b> 1-800-267-4414</p> <p><b>Fax On Demand</b> 1-800-646-1655</p> <p><b>Website</b> <a href="http://www.3M.com/CA/OccSafety">http://www.3M.com/CA/OccSafety</a></p> <p><b>Technical Assistance In Mexico</b> 01-800-712-0646 5270-2255, 5270-2119 (Mexico City only)</p> <p><b>Technical Assistance In Brazil</b> 0800-132333</p> <p><b>Fax On Demand O.U.S. Locations</b> 1-651-732-6530</p>
---