Wear It Right

Wearing your adjustable strap valved filtering facepiece respirator


1 Place the respirator over your nose and mouth. Be sure the metal nose clip is on top.

2 Pull the bottom strap over your head until it rests below the ears at the back of your neck.

3 Pull the top strap over your head until it rests on the crown of your head above your ears.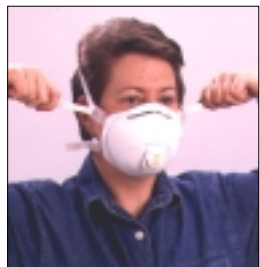

4 Without removing the respirator, adjust the straps until the respirator is secure. To tighten the respirator, gently pull the ends of both straps. To loosen the respirator, press on the back side of the buckle.

5 Using both hands starting at the top, mold the metal nose clip around your nose to achieve a secure seal.

Adjustable strap valved filtering facepiece respirators

Check the seal of your adjustable strap valved filtering facepiece respirator each time you don the respirator.

Place both hands over the respirator and *inhale* sharply. The respirator should collapse slightly. If air leaks between the face and facesal of the respirator, reposition it and adjust the nose clip for a more secure seal. If you cannot achieve a proper seal, **do not** enter the contaminated area. See your supervisor.

3M

General Offices
3M Center
St. Paul, MN 55144-1000

 **WARNING:**

- ▲ Breathing certain dusts, mists, fumes, gases or vapors can cause sickness or death.
- ▲ This respirator can help protect you when worn during all times of exposure.
- ▲ If you cannot achieve a proper fit, do not enter the contaminated area. See your supervisor.
- ▲ See your supervisor with questions regarding which respirator to wear and its proper use, or call 3M: 1-800-243-4630 Internet: www.3m.com/occsafety E-mail: occsafety@mmmm.com (In Canada, call 3M: 1-800-267-4414, E-mail: ohes@ca.mmm.com).
- ▲ OSHA's standard 1910.134 and the CSA standard Z94.4-93 requires that the wearer be fit tested.
- ▲ Do not use with beards or other facial hair or other conditions that prevent a good seal between the face and the facesal of the respirator.

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© 3M 2000 70-0707-9707-4