# ÚSELO BIEN

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**3M** Innovación

# Wear It Right

## Wearing your three-panel filtering facepiece respirator

**1** Remove the respirator from its packaging and hold with straps facing upward. Place the bottom strap under the center flaps next to the "WARNING" statement.

**2** Fully open the top and bottom panels, bending the nosepiece around your thumb at center of the foam. Straps should separate when panels are opened. Make certain the bottom panel is unfolded and completely opened.

**3** Place the respirator on your face so that the foam rests on your nose and the bottom panel is securely under your chin.

**4** Pull the top strap over your head and position it high on the back of the head. Then, pull the bottom strap over your head and position it around your neck and below your ears.

**5** Adjust for a comfortable fit by pulling the top panel toward the bridge of your nose and positioning the bottom panel under your chin.

**6** Place fingertips from both hands at the top of your nose and mold the nosepiece around your nose to achieve a secure seal.

Non-valved, three-panel filtering facepiece respirator


Valved, three-panel filtering facepiece respirator

## Check the seal of your three-panel facepiece respirator each time you don the respirator.

**1 For non-valved respirators** Place one or both hands completely over the middle panel. *Inhale and exhale sharply.* If air leaks around your nose, readjust the nosepiece. If air leaks between the face and face seal of the respirator, reposition it by adjusting the panels and straps. If you cannot achieve a proper seal, **do not** enter the contaminated area. See your supervisor.

**2 For valved respirators** Place one or both hands completely over the middle panel and *inhale* sharply. If air leaks around your nose, readjust the nosepiece. If air leaks between the face and face seal of the respirator, reposition it by adjusting the panels and straps. If you cannot achieve a proper seal, **do not** enter the contaminated area. See your supervisor.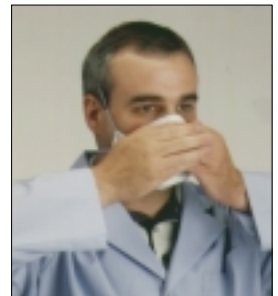

**WARNING:**

- ▲ Breathing certain dusts, mists, fumes, gases or vapors can cause sickness or death.
- ▲ This respirator can help protect you when worn during all times of exposure.
- ▲ If you cannot achieve a proper fit, do not enter the contaminated area. See your supervisor.
- ▲ See your supervisor with questions regarding which respirator to wear and its proper use, or call 3M: 1-800-243-4630. Internet: [www.3m.com/occsafety](http://www.3m.com/occsafety) E-mail: [occsafety@mmm.com](mailto:occsafety@mmm.com) (In Canada, call 3M: 1-800-267-4414, E-mail: [ohes@ca.mmm.com](mailto:ohes@ca.mmm.com)).
- ▲ OSHA standard 1910.134 and the CSA standard Z94.4-93 requires that the wearer be fit tested.
- ▲ Do not use with beards or other facial hair or other conditions that prevent a good seal between the face and the face seal of the respirator.

General Offices  
3M Center  
St. Paul, MN 55144-1000

**3M Innovation**