

3M Unitek

SmartClip™ Appliance Wire Insertion Instrument Instructions



SMARTCLIP™
SELF-LIGATING APPLIANCE SYSTEM

3M

3M Unitek
Orthodontic Products
2724 South Peck Road
Monrovia, CA 91016 USA

CE

European Representative:

3M Unitek
Ohmstrasse 3
86899 Landsberg, Germany
+49/(0)8191/ 9474-0

Dentronix is a registered trademark of Dentronix, Inc.
Steril-Quik is a trademark. SmartClip is a trademark of 3M.

© 2006, 2009 3M. All rights reserved. REF 011-616-2 0907


SmartClip™ Appliance Wire Insertion Instrument Instructions

Instructions For Use


English




Single-handed technique

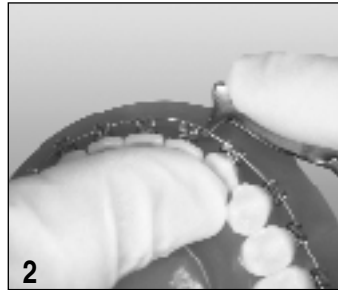

Dual-handed technique

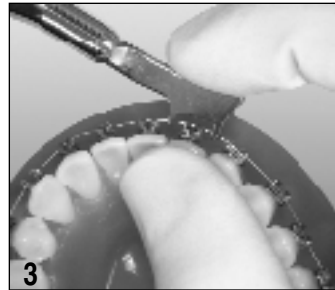

Single-handed technique

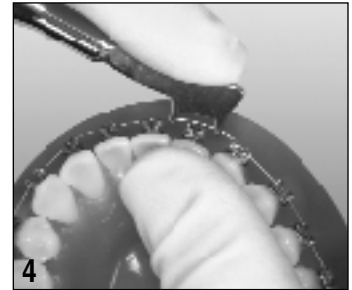

Dual-handed technique

Technique using SINGLE POINT END:

1. Align archwire over clip opening and bracket slot.
2. Position instrument on archwire on either mesial or distal side of bracket. (Figures 1 & 2)
3. Torque the archwire if necessary.
4. Push instrument gently while providing lingual support to the teeth.

Note: Repeat procedure for each bracket.

Technique using TORQUING KEY END:

1. Align archwire over clip opening and bracket slot.
2. Position instrument on archwire so that torquing key straddles the bracket. (Figures 3 & 4)
3. Torque the archwire if necessary.
4. Push instrument gently while providing lingual support to the teeth.

Note: Repeat procedure for each bracket.

WARNINGS:

Use instruments for indicated use only. Failure to do so may result in the breakage of the instrument and injury to patient or orthodontic professional. Safety glasses recommended for patient and orthodontic staff when using these instruments.

This product contains nickel and/or chromium. A small percentage of the population is known to be allergic to nickel and/or chromium. If an allergic reaction occurs, direct patient to consult a physician.

Disinfection Information

Please refer to literature 011-650, "Reprocessing Instructions For Use".

Warranty Information

3M Unitek warrants that this instrument will be free from defects in materials, or workmanship for a period of 1 year from the date of purchase. Failure to follow recommended use, maintenance and sterilization procedures will void all warranty claims.

Recommended maintenance includes cutter tip/blade re-sharpening (if applicable) and reconditioning (cleaning, polishing, adjusting screw as necessary) every six months. 3M Unitek's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to, at 3M Unitek's option, repair, replacement of merchandise, or refund of the purchase price. ALL IMPLIED WARRANTIES, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE OR USE ARE DISCLAIMED.

Limitation of Liability

Except where prohibited by law, 3M Unitek will not be liable for any loss or damages arising from this Product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

All instruments should be sent to the 3M Unitek Authorized Service Center at the address below for evaluation to determine credit, replacement, or repair. The Authorized Service Center will determine, upon receipt of instrument, if the instrument is to be covered by warranty. Materials, workmanship, and corrosion resistance are the only items covered by this warranty. Misuse or abuse will void warranty. Customer will be contacted regarding repair charges for out of warranty instruments.

3M Unitek – Authorized Service Center

Precision Plier Service
14175 Telephone Avenue, Suite D
Chino Hills, CA 91710-5762
Phone Number: 1-800-229-9060 USA, Canada. 1-909-590-2085.

References

ISO 17664:2004(E), Sterilization of medical devices—Information to be provided by the manufacturer for the processing of resterilizable medical devices.