

# 3M™ Cavilon™ Skin Protection Products

Use them  
from  
to  
to  
head  
to  
toe

On surgical incisions  
(3M™ Cavilon™ No Sting Barrier Film only)

On minor burns  
(Cavilon™ No Sting Barrier Film only)

Underneath ostomy  
appliances and surrounding  
skin to protect

To protect hands  
from frequent  
hand washing and  
dryness from gloves

For skin irritation  
from incontinence

For areas of chafing

To protect skin  
under a prosthesis

On scrapes or cuts  
(Cavilon™ No Sting Barrier Film only)

On skin tears  
(Cavilon™ No Sting Barrier Film only)

Under any adhesive tape  
to prevent tape sensitivities

On heels and elbows  
to prevent breakdown  
from friction



# 3M™ Cavilon™ Skin Protection Products



Use them  
from  
to  
to  
head  
to  
toe



**3M Health Care**  
**3M Canada**  
P.O. Box 5757  
London, Ontario N6A 4T1  
1-800-364-3577  
[www.3M.com/ca/healthcare](http://www.3M.com/ca/healthcare)

**3M Health Care**  
3M Centre, Building 275-4W-02  
St. Paul, MN 55144-1000  
USA

Please recycle. Printed in Canada.  
© 3M 2007. All rights reserved.  
3M and Cavilon are trademarks of 3M. Used under license in Canada 0703-MS-28200-E



# 3M™ Cavilon™ Skin Cleanser

Cleansing is the important first step in managing your skin. Cavilon™ Skin Cleanser is an excellent choice for all your skin cleansing needs, including incontinence skin care.



## Key Features

- Can be used on intact or irritated skin
- pH balanced
- Contains ingredients to condition skin
- Gentle – cleans without irritation
- Helps control odour
- Convenient, no-rinse formula
- Compatible with other Cavilon™ products
- Contains an emollient (agent that softens and soothes skin when applied topically)

## Directions For Use

1. Moisten a clean cloth with Cavilon™ Skin Cleanser or water.
2. Spray affected area with Cavilon™ Skin Cleanser.
3. Gently wipe skin with the clean cloth to remove urine, stool or soil. Rinsing is not needed. Repeat as needed.

# 3M™ Cavilon™ Durable Barrier Cream

Protecting intact skin that may be prone to breakdown is critically important. Cavilon™ Durable Barrier Cream is the next step to ensuring your skin stays as healthy as possible.



## Key Features

- Moisturizes (imparts or restores moisture to the skin)
- Contains an emollient (softens and soothes skin)
- Forms a highly effective protective coating on the skin
- Repels moisture and irritants
- Skin protectant
- Does not clog briefs
- Concentrated
- Does not interfere with tape adhesion
- Resists wash-off

## Directions For Use

1. Cleanse skin with Cavilon™ Skin Cleanser, then pat dry.
2. Apply Cavilon™ Durable Barrier Cream sparingly to cover entire affected area\*.
3. Repeat as necessary, every 4-6 washings. (If the feel is oily, too much has been applied.)

\* Note: Can only be used on healthy, intact skin.

# 3M™ Cavilon™ No Sting Barrier Film

Managing irritated skin that is prone to breakdown or incontinence dermatitis is crucial. Cavilon™ No Sting Barrier Film forms a transparent, protective film between the skin and irritants (urine, feces or adhesives).



## Key Features

- Resists wash-off (will last up to 72 hours for incontinence care)
- Alcohol-free (will not cause pain even when applied to irritated or damaged skin)
- Forms a highly effective protective coating on the skin
- Does not clog briefs
- Hypoallergenic
- Transparent
- Quick drying
- Protects intact or damaged skin from friction, adhesives and body fluids

## Directions For Use

1. Skin should be clean and dry prior to application of Cavilon™ No Sting Barrier Film.
2. Apply a uniform coating of barrier film over entire area of concern with the foam wipe. If using the spray bottle, hold nozzle 4-6 inches from skin and spray in a sweeping motion.
3. If Cavilon™ No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, ensure that these areas of skin are separated to allow coating to dry thoroughly before allowing skin to return to normal position.
4. Reapplication is recommended every 24-72 hours depending on cleansing frequency. (If build-up or flaking occurs, too much has been applied.)

Use them  
from head  
to toe

