

3M™ Surgical Clipper 9671

For use with 9670 blade



Economical, reliable clipper streamlines patient preparation

- Ergonomic, non-slip handle fits comfortably in hand
- Away-from-the-skin blade positioning minimizes potential for nicks or cuts
- Blade snaps on and off easily
- Clean by rinsing or wiping with disinfectant
- Operates with or without cord
- Provides 45 minutes of cordless run time on a 12-14 hour recharge
- Has a nick rate of <1%, compared with other clippers' higher nick rates*

Ordering Information

Catalog Number	Product Description	Items/Box	Boxes/Case
9670	3M™ Surgical Clipper Blade 9670	50	1
9671	3M™ Surgical Clipper 9671	1	1
9672	3M™ Surgical Clipper Charger/Cord 9672	1	1
9676	3M™ Surgical Clipper Charger Stand 9676	1	1
9677	3M™ Surgical Clipper Starter Kit 9677: 3M™ Surgical Clipper 9671 and 3M™ Surgical Clipper Charger/Cord 9672 (clipper blade and charge stand not included)	1	1
9669	3M™ Surgical Clipper Wall Bracket 9669	1	1

For more information about the 3M™ Surgical Clipper 9671 or other 3M Health Care products:

- Contact your local 3M sales representative
- Call the 3M Health Care Customer Helpline at 1-800-228-3957
- Visit 3M.com/infectionprevention



How to use

3M™ Surgical Clipper 9671

For use with 9670 blade

Changing Blades



Attach

Remove backing from the blade packaging, hold clipper body at a 45-degree angle and snap on single-use blade 9670.

NOTE: An audible snap indicates the blade is securely attached.



Remove

To remove the blade, use your thumb to “pop off” the used blade into a sharps waste receptacle.

NOTE: Always use a new blade for each patient to avoid cross-contamination.

Clipping



Push

Position the clipper like a pencil with the clipper blade flat or at a slight angle on the skin surface. Gently push the clipper forward, against the direction of the patient's hair growth.

NOTE: Do not use excessive pressure to push the clipper down on the patient's skin.



Pull

Position the blade flat or at a slight angle on the skin surface and gently pull the clipper toward you.

NOTE: Do not rake the blade across the patient's skin.

Cleaning



Rinse

With the clipper uncorded and in the “off” position and the blade removed, rinse the handle under warm running water. Dry thoroughly before recharging or reusing.

NOTE: Do not submerge clipper body in any liquid solution



Wipe

With the clipper in the “off” position and the blade removed, wipe the clipper handle with an alcohol or disinfectant wipe.

Charging



Charge

Before initial use, charge clipper for 18-24 hours. After each use, store the clipper handle in the “off” position in the plugged-in charge stand. If not using the charge stand, plug the cord into the bottom of the clipper. Clipper provides approximately 45 minutes of cordless run time on a 12-14 hour recharge or has continuous use when operating with the cord.

NOTE: There is no risk of overcharging, but after the initial use, charging for more than 14 hours will not result in longer run time.

Refer to package insert for warnings and more detailed instructions.

3M

**3M Health Care
Infection Prevention Division**
3M Center, Building 275-4E-01
St. Paul, MN 55144-1000
U.S.A.
800-228-3957
3M.com/infectionprevention

**Medical Division
3M Canada**
Post Office Box 5757
London, Ontario N6A 4T1
Canada
800-563-2921

3M is a trademark of 3M.
Please recycle. Printed in U.S.A.
© 3M 2012. All rights reserved.
70-2010-9136-3

