

Instructions for Use for Open Eye Technique



1 Prep and thoroughly dry lids and lashes.



2 Assistant to hold lid open with cotton tip applicators.



3 Position and center incise area on eye.

4 Withdraw applicators.



5 Unfold drape.

- Gently press incise drape around bony orbit
- Grasp film with smooth-tipped forceps


6 Cut through incise slightly below midline.

7 Insert speculum, wrapping incise plastic around lid margin.

3M™ Steri-Drape™ Ophthalmic Incise Drapes

Ordering Information

Catalog Number	Drape Dimensions	Adhesive Patch or Incise Dimensions	Pouch	Drapes/Box	Boxes/Case
1021	15" x 15" 40cm x 40cm	3 7/8" x 4 7/8" 10cm x 12.5cm	–	10	4
1023	15" x 19" 40.5cm x 48cm	4 7/8" x 3 7/8" 12.5cm x 10cm	Single	10	4
1055B	62" x 108" 158.8cm x 274.3cm	3 1/2" x 4 5/8" 9cm x 11.7cm	Double	20	1
1060	51" x 51" 130cm x 130cm	3 7/8" x 4 7/8" 10cm x 12.5cm	–	10	4
1061	51" x 51" 130cm x 130cm	3 7/8" x 4 7/8" 10cm x 12.5cm	Single	10	4
1062	53" x 53" 137cm x 137cm	3 1/8" x 3 7/8" 8cm x 10cm	Single	20	1
2035	7 1/8" x 7 1/8" 20cm x 20cm	3 7/8" x 7 7/8" 10cm x 20cm	–	10	4

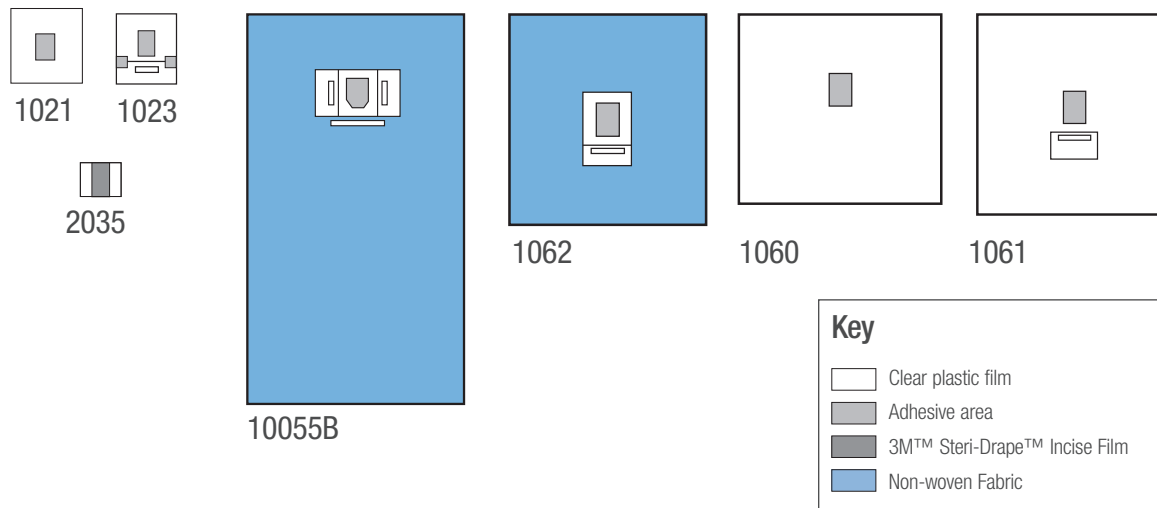

For more information about 3M™ Surgical Drapes or other 3M Health Care products:

- » Contact your local 3M representative
- » Call the 3M Health Care Customer Helpline at 800-228-3957
- » Visit www.3M.com/infectionprevention



Infection Prevention Division 3M Health Care

2510 Conway Avenue
St. Paul, MN 55144-1000
U.S.A.
1-800-228-3957
www.3M.com/infectionprevention

3M and Steri-Drape are trademarks of 3M.
©3M 2014. All rights reserved.
70-2011-5605-9