

# Instructions for Use for Small Aperture Drapes



**1** Prep and thoroughly dry lids and lashes.



**2** Unfold drape.

- Assistant to hold lower edge taut, not touching the face
- Surgeon to grasp drape on sterile surface with index fingers and thumbs



**3** Position edge of aperture approximately 5 mm below upper lid margin.

- Press on lashes and lid margin
- Draw, drag or pull lid open
- Secure adhesive by pressing down

**4** Repeat with lower lid.

**5** Adjust any lateral wrinkles in drape.

**6** Assess exposure.

**7** Apply speculum.


# 3M™ Steri-Drape™ Ophthalmic Aperture Drapes

## Ordering Information


Catalog Number	Drape Dimensions	Adhesive Patch or Incise Dimensions	Aperture/Shape Dimensions	Pouch	Drapes/Box	Boxes/Case
1020	15" x 15" 40cm x 40cm	3 7/8" x 4 7/8" 10cm x 12.5cm	Circle 2 3/8" 6.3cm	–	10	4
1024	15" x 19" 40.5cm x 49cm	3 7/8" x 4 7/8" 10cm x 12.5cm	Oval 1 3/8" x 2 3/8" 3.5cm x 6.3cm	Single	10	4
1030	31" x 51" 80cm x 130cm	3 7/8" x 4 7/8" 10cm x 12.5cm	Oval 2" x 2 7/8" 5.2cm x 7.6cm	–	10	4
1022	15" x 10" 40cm x 27 cm	2-3/4" x 2-3/4" 7,2cm x 7,2cm	Oval 1-3/4" x 1-1/2" 4,5cm x 3,8cm	–	20	4
1064	62" x 72" 158cm x 182cm	3-1/2" x 4-1/2" 9cm x 11,7cm	2-1/8" x 1-1/8" 5,7cm x 2,9cm	Two	20	2
1065B	62" x 108" 158cm x 274cm	3-1/2" x 4-1/2" 9cm x 11,7cm	2-1/8" x 1-1/8" 5,7cm x 2,9cm	Two	14	2




1020

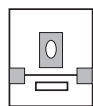


1024

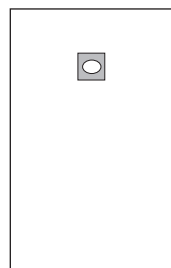


1030

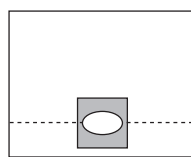

1022

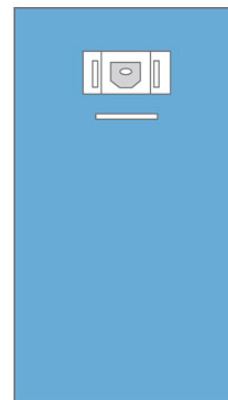

1065B

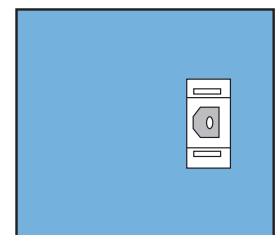

1064

### Key

	Clear plastic film
	Adhesive area
	Non-woven Fabric

For more information about 3M™ Surgical Drapes or other 3M Health Care products:

- » Contact your local 3M representative
- » Call the 3M Health Care Customer Helpline at 800-228-3957
- » Visit [www.3M.com/infectionprevention](http://www.3M.com/infectionprevention)



### Infection Prevention Division 3M Health Care

2510 Conway Avenue  
St. Paul, MN 55144-1000  
U.S.A.  
1-800-228-3957  
[www.3M.com/infectionprevention](http://www.3M.com/infectionprevention)

3M and Steri-Drape are trademarks of 3M.  
©3M 2014. All rights reserved.  
70-2011-5604-2