




3M™ Tegaderm™ I.V. Advanced Securement Dressing 1689

Application & Removal Guide for PICC Catheters

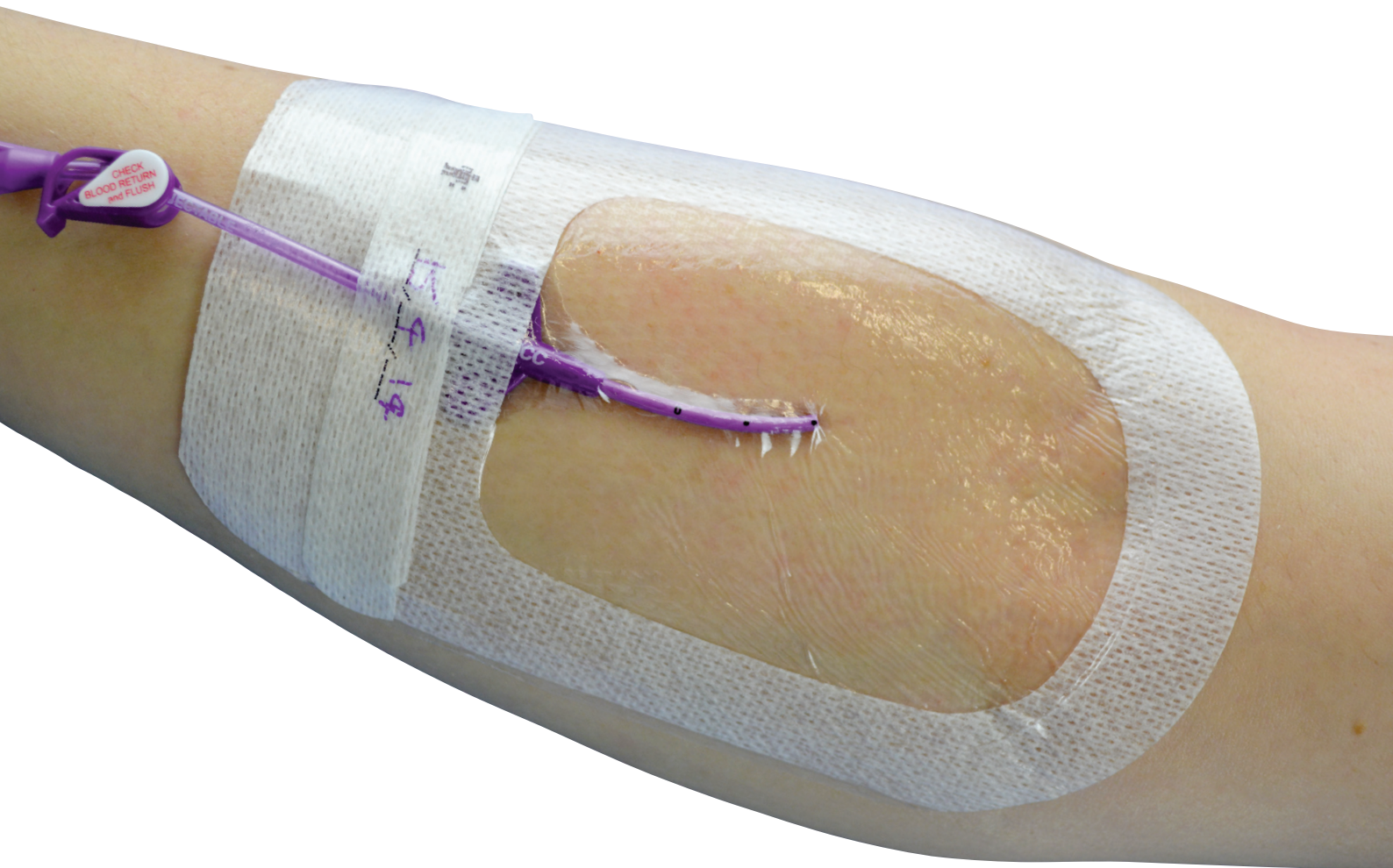



3M™ Tegaderm™ I.V. Advanced Securement Dressing for PICC Application

APPLICATION



1. Prepare the site according to your facility's protocol. Allow all preps and skin protectants to dry completely. Open package and place dressing onto sterile field.



2. Once ready to use, remove the liner to expose adhesive. Flip over dressing so adhesive faces skin.



3. Position notch edge of dressing over catheter hub, placing stabilisation border over catheter hub "wings". Apply dressing to skin (ensure you are not stretching the dressing over the skin surface during application).




4. Pinch film firmly around catheter insertion site and along to the catheter hub. Gently lift the catheter lumens and press border tabs firmly under catheter hub and lumens. Apply firm, gentle pressure to entire window film area.



5. Slowly peel frame away from border edge while simultaneously applying firm pressure to border edge. Smooth entire dressing onto skin to ensure optimal adhesion.



6. Remove securement tape strip with notch from paper frame. Two small adhesive free tabs from the frame will remain on back of tape strip. Utilising these adhesive free areas to place your fingers, partially fold the securement tape strip in half at notch, while using your other hand to hold up the catheter lumens

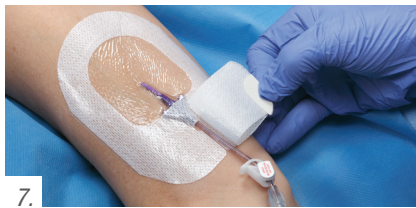

7. Apply the tape strip's pre-cut notch completely underneath the catheter's hub area and lumens (but over the top of the dressing). The notch opening should face the insertion site. Peel paper tabs from back of tape strip and smooth the entire tape surface working from the notched area towards the dressing edge



8. Prepare and apply documentation label over the top of the exposed moulded junction. Press the first half of the strip against the junction; then adhere the other side.



9. Apply firm but gentle pressure to entire dressing to ensure optimal adhesion.

REMOVAL



1. Remove all securement tape strips from dressing.



2. Gently grasp border tabs below keyhole notch and gently start peeling them back while stabilising the catheter.



3. Continuing to stabilise the catheter and patient skin, carefully peel the dressing back toward the catheter insertion site until removed using the low and slow technique.

4. Remove securement device as per facility policy.

3M Critical & Chronic Care Solutions Division

3M Australia Pty Limited
ABN 90 000 100 096
Building A, 1 Rivett Road
North Ryde NSW 2113
1300 363 878
www.3M.com.au/healthcare

3M New Zealand Limited
94 Apollo Drive
Rosedale,
Auckland 0632
0800 80 81 82
www.3M.co.nz

3M and Tegaderm are trademarks of 3M Company. Please recycle.
Printed in Australia. © 3M 2014. All rights reserved. PB6727/0514

Optional-Use with securement device

1. Prepare the site according to your facility's protocol. Allow all preps and skin protectants to dry completely. Open package and place dressing onto sterile field.
2. Place securement device onto patient skin and secure catheter as per hospital policy.
3. Remove dressing liner and ensure dressing is positioned to completely incorporate the securement device.
4. Carry out steps 3 to 9 above.

