## **3M ESPE Temporization Solutions**



# A simple method for shine everytime.



# 3M ESPE makes it easy to create precise provisionals with shine everytime.



#### It's as simple as 1, 2, 3.

Clinical photos courtesy of Dr. Gunnar Reich

Take the preliminary impression using Imprint<sup>™</sup> 4 Preliminary VPS Impression Material.



Fill the preliminary impression with Protemp<sup>™</sup> Plus Temporization Material and seat in the mouth. Hold firmly in place for at least 1:40 minutes from start of mix.

To ensure a smooth and shiny surface, leave the material in the matrix for 5 minutes. No polishing is needed. Simply wipe with alcohol to reveal the shine.







## Pentamix<sup>™</sup> Lite Automatic Mixing Unit

When you're ready to take the preliminary impression simply start with the press of a button.

# Start automixing and take control.

Pentamix Lite automatic mixing unit delivers a homogeneous, void-free mix in the exact quantity you need.





reddot award 2014 winner

## Use Only the Original!

Always use **3M ESPE's Penta<sup>™</sup> Mixing Tips** in your Pentamix device to ensure proper mixing and impression results.

## Another plus: 3M<sup>™</sup> ESPE<sup>™</sup> Impression Tray.

With no need for a tray adhesive, the single-use **3M ESPE Impression Tray** is ready to use and saves time. Simply select the proper tray size (S, M or L) and fill the tray directly from the mixing device.



# Imprint<sup>™</sup> 4 Preliminary

VPS Preliminary Impression Material

No dust. No mess. Gain control with stress-free preliminary impressions. Once you try this VPS material, alginate will be a thing of the past.

## A smooth impression makes provisionals shine.

Over 95% of dentists surveyed noticed a better surface quality of Protemp<sup>™</sup> Plus temporary restorations when the matrix was made with Imprint 4 Preliminary VPS Material vs. an alginate material.<sup>1</sup>

## Patient-pleasing mint flavor. Available in two setting versions, regular and super quick.

### Smooth impressions mean less work for you.



Alginate surface (Palgat<sup>™</sup> Plus, 3M ESPE) shown with a Laser Scanning Microscope (objective lens with 20x magnification).



Surface of Imprint<sup>™</sup> 4 Preliminary VPS Material shown with a Laser Scanning Microscope (objective lens with 20x magnification).

Source: 3M ESPE internal data.

# A backup for peace of mind.

1 Source: Field evaluation conducted by 3M ESPE.

Impressions show virtually no dimensional change—even after three months. This allows for easy remakes of provisionals if necessary.

## **Protemp<sup>TM</sup> Plus** Temporization Material

Get an easy shine every time when your Protemp<sup>™</sup> Plus provisional is made with Imprint<sup>™</sup> 4 Preliminary VPS Impression Material.

Record-breaking toughness. Available in six shades.

ation Material

67g Paste

## The high-performance material for all impression-based provisionals.

You and your patients will benefit from a reliably tough temporary that looks great without any additional polishing or glaze. Simply wipe with alcohol for a natural shine.

## **Ordering Information**

PENTAMIX <sup>™</sup> LITE AUTOMATIC MIXINO	UNIT AND	ACCESSORIES
---	----------	-------------

Item No.	Product Information
77907	Pentamix <sup>™</sup> Lite Automatic Mixing Unit – 120 V/60 Hz Includes: 1 Penta <sup>™</sup> Cartridge for Pentamix <sup>™</sup> Lite
77944	1 Penta™ Cartridge for Pentamix™ Lite
77949	Penta™ Mixing Tips – Red Includes: 50 Mixing Tips

3M<sup>™</sup> ESPE<sup>™</sup> IMPRESSION TRAY REFILLS (each containing 10 trays)

Product Information
Size S, upper jaw
Size M, upper jaw
Size L, upper jaw
Size S, lower jaw
Size M, lower jaw
Size L, lower jaw

#### IMPRINT<sup>™</sup> 4 PRELIMINARY PENTA<sup>™</sup> VPS PRELIMINARY IMPRESSION MATERIAL

Item No.	Product Information	
	Refills	
71521	Imprint <sup>™</sup> 4 Preliminary Penta <sup>™</sup> Includes: 2 base pastes – 300 ml each, 2 catalysts – 60 ml each	
71522	Imprint <sup>™</sup> 4 Preliminary Penta <sup>™</sup> Super Quick Includes: 2 base pastes – 300 ml each, 2 catalysts – 60 ml each	
	Value Packs	
71523	Imprint <sup>™</sup> 4 Preliminary Penta <sup>™</sup> Includes: 6 base pastes – 300 ml each, 6 catalysts – 60 ml each	
71524	Imprint <sup>™</sup> 4 Preliminary Penta <sup>™</sup> Super Quick Includes: 6 base pastes – 300 ml each, 6 catalysts – 60 ml each	

#### IMPRINT<sup>™</sup> 4 PRELIMINARY VPS PRELIMINARY IMPRESSION MATERIAL

Product Information
Economy Packs
Imprint™ 4 Preliminary Regular Includes: 10 – 50 ml cartridges
Imprint <sup>™</sup> 4 Preliminary Super Quick Includes: 10 – 50 ml cartridges
Bulk Packs
Imprint™ 4 Preliminary Regular Includes: 30 – 50 ml cartridges
Imprint <sup>™</sup> 4 Preliminary Super Quick Includes: 30 – 50 ml cartridges

#### **PROTEMP<sup>™</sup> PLUS TEMPORIZATION MATERIAL**

ltem No.	Product Information	
46962	Introductory Kit – A2 Includes: 1 cartridge – 50 ml, A2; 16 Garant <sup>™</sup> Mixing Tips blue; 1 Garant <sup>™</sup> Dispenser 10:1	
	Refills	
46964	Protemp <sup>™</sup> Plus Temporization Material – Bleach Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
46955	Protemp <sup>™</sup> Plus Temporization Material – A1 Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
46963	Protemp <sup>™</sup> Plus Temporization Material – A2 Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
46958	Protemp <sup>™</sup> Plus Temporization Material – A3 Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
46973	Protemp <sup>™</sup> Plus Temporization Material – A3.5 Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
46961	Protemp <sup>™</sup> Plus Temporization Material – B3 Includes: 1 cartridge – 50 ml, 16 Garant <sup>™</sup> Mixing Tips blue	
ltem No.	Product Information	
71/52	Coront™ Mixing Tine Defill Dlue	

71453	Garant™ Mixing Tips Refill — Blue
	Includes: 50 Mixing Tips



#### **Dental Products**

2510 Conway Avenue St. Paul, MN 55144-1000 USA 1-800-634-2249

#### 3M Canada

Post Office Box 5757 London, Ontario N6A 4T1 Canada 1-888-363-3685

3M, ESPE, Garant, Imprint, Palgat, Penta, Pentamix and Protemp are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2014. All rights reserved. Please recycle. Printed in U.S.A. 70-2013-0664-7 (Rev. B)