

Trusted Connection to Dental Wings Software



3M™ Margin Marking Software gives lab technicians a unique view of the oral anatomy, allowing them to accurately identify and mark the margin line on the 3D model. The margin line, 3D model and prescription information can then be easily converted and imported into Dental Wings. Affordable, subscription-based Lab Packages give labs access to the 3M™ Connection Center and the digital impression files they need. Access to this Trusted Connection requires a subscription to the Standard Lab Package.




How does it work?



1. Scan and Send

Doctors scan using the 3M™ True Definition Scanner to create a true 3D replica of the oral anatomy. The accuracy of the 3M™ True Definition Scanner provides a solid foundation for your lab to create high quality, precisely fitting restorations. Cases are sent directly from the scanner to the 3M™ Connection Center – a secure digital hub that stores, sends and converts the digital impression files.

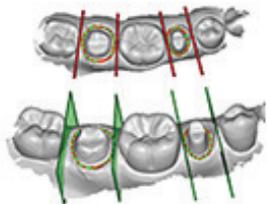

2. Margin Mark and Convert

Labs can download the case to the 3M™ Margin Marking Software to accurately mark the margin. The files are sent back to 3M™ Connection Center and converted to the appropriate file type for Dental Wings. The file includes the prescription information, margin line and 3D scan files necessary for design.



3. Download and Import

Open the inbox in your Dental Wings software. Cases sent to your lab through the 3M™ Connection Center will automatically be made available to you in the inbox.



4. Design and Fabricate

The Dental Wings software opens up a world of design and fabrication options for doctors and labs. Dental Wings offers modules for full contour and cutback restoration design, implant supported prostheses and a variety of orthodontic and dental appliances giving clinicians and lab technicians access to a wide variety of solutions.

How do I get started?

Download the model builder module and license from Dental Wings. You will also need to enable your Dental Wings inbox to access the 3M™ Connection Center. Once enabled, this connection will allow you to import cases directly to your machine. This import functionality works on Dental Wings software version 3.6 and higher. Your lab will also need to be enabled to receive Dental Wings files from the 3M™ Connection Center. Please contact 3M Customer Care 1-800-634-2249, Option 3, or email customercaresoc@mmm.com for more information.



3M ESPE Dental
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249
Web 3m.com/truedef

Please recycle. Printed in USA © 3M 2015.
All rights reserved. 3M and ESPE are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Dental Wings is a trademark of Dental Wings.