


Supplies You May Need

- Gloves
- Saline or irrigation solution
- Gauze pads
- Clean towels or underpads to protect patient and surfaces
- **3M™ Tegaderm™ Silicone Foam Border Dressing**
Select a dressing size to allow for 2.5 cm (1 inch) of foam to extend beyond the wound edge

- **3M™ Cavilon™ No Sting Barrier Film**

Indications for Use: Designed as a primary or secondary dressing for management of low- to highly exudating, partial and full thickness wounds such as pressure ulcers, venous leg ulcers, neuropathic ulcers, arterial ulcers, skin tears, surgical wounds, superficial partial thickness burns, and donor sites. This dressing is suitable for use on fragile skin and with compression therapy.

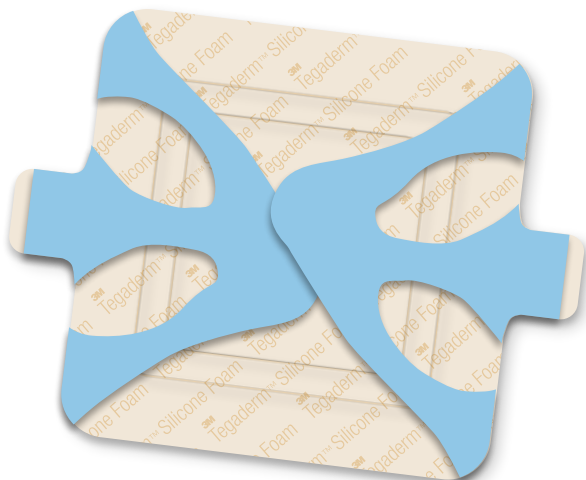

Ordering Information

3M™ Tegaderm™ Silicone Foam Border Dressing

	Product Code	Overall Dressing Size	Foam Pad Size	Dressings/Box	Boxes/Case
	90640	3 in. x 3 in. 7.5 cm x 7.5 cm	1½ in. x 1½ in. 3.7 cm x 3.7 cm	10	6
	90641	4 in. x 4 in. 10 cm x 10 cm	2⅞ in. x 2⅞ in. 5.9 cm x 5.9 cm	10	6
	90642	6 in. x 6 in. 15 cm x 15 cm	4⅞ in. x 4⅞ in. 10.3 cm x 10.3 cm	10	4

For more information about the 3M™ Tegaderm™ Foam Family of Products, visit www.3m.ca/TegadermFoam, contact your 3M Critical and Chronic Care Solutions Representative, or call the 3M Health Care Customer Helpline at **1-800-364-3577**.

3M™ Tegaderm™ Silicone Foam Silicone Foam Border Dressing



Application and Removal Pocket Guide

3M, Cavilon and Tegaderm are trademarks of 3M. Used under license in Canada. Please recycle. Printed in Canada. © 2013, 3M. All rights reserved. 1304-01069



Available in Canada from:

3M Critical & Chronic Care
Solutions Division
P.O. Box 5757
London, ON N6A 4T1
1-800 3M HELPS
www.3m.ca


3M Health Care
2510 Conway Avenue
St. Paul, MN, US, 55144



Prior to Application

Follow Universal Precautions and facility guidelines for infection control. Wash or sanitize hands, set up a clean field with supplies, apply gloves.

Cleanse the wound and periwound skin according to your institution's protocol. Dry the area around the wound with dry gauze.

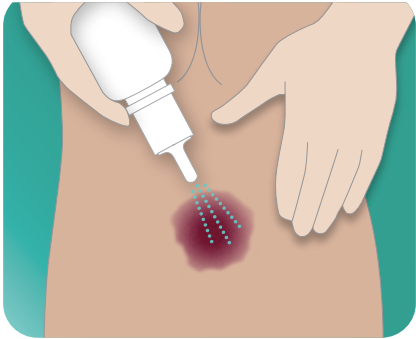

Precaution: Do not use the dressing in combination with oxidizing agents such as hydrogen peroxide or hypochlorite solution. These solutions may affect the structure and performance of the polyurethane foam.

Assess the wound and surrounding area for signs of infection. If infection is present, consult a health care professional. When used on infected wounds, 3M™ Tegaderm™ Silicone Foam Border Dressing may only be applied under the supervision of a health care professional.

Remove unclean gloves and wash or sanitize hands. Apply new gloves.


Note: If periwound skin is fragile or exposure to wound exudate is likely, apply a barrier film, such as 3M™ Cavilon™ No Sting Barrier Film. Let dry.

Directions for Use


3M™ Tegaderm™ Silicone Foam Border Dressing




1. Hold the dressing by one of the side tabs and remove the liner.

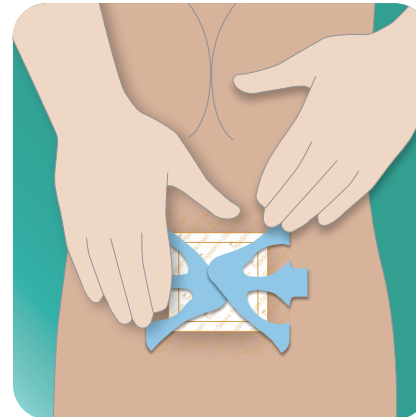


2. While holding the tabs, position the dressing over the wound. Gently secure the dressing to the skin.

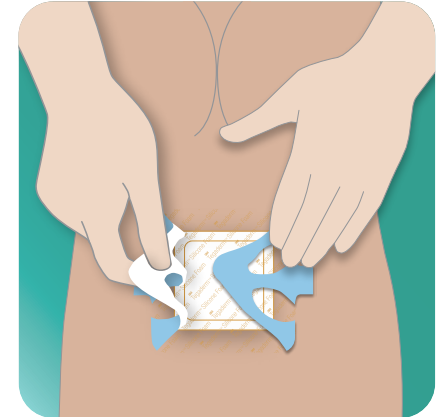

3. Starting at the center, lift the blue application tabs off the dressing. Repeat this for the remaining section. Avoid stretching the dressing or the skin.

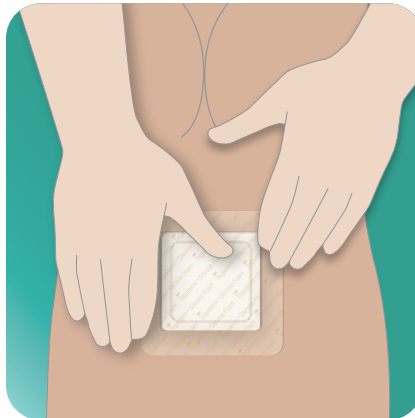

4. Apply pressure to the dressing to conform to the skin. The translucent film helps make wound care discreet and the silicone printing makes identification clear to clinicians.

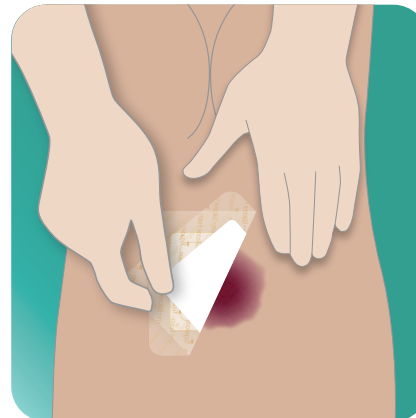

5. The Tegaderm™ Silicone Foam Border Dressing has a special adhesive which allows for repositioning.

Dressing removal

Tegaderm™ Silicone Foam Border Dressing may remain in place for up to seven days. Dressing change frequency will depend on the type of wound, volume of exudate and clinical situation.

When the exudate spreads to the edges of the foam pad, or the dressing begins to leak, a dressing change is indicated. To remove dressing, carefully lift the corners of the dressing from the skin. (If there is difficulty lifting the film border, apply tape to the edge of the dressing and use the tape to lift). Continue lifting the edges until all are free from the skin surface. Remove the dressing.

Important Note: Refer to the product insert for complete information related to the use of Tegaderm™ Silicone Foam Border Dressing. This application guide is intended only as a quick reference.