




Custom Application Using Mini Wrap*

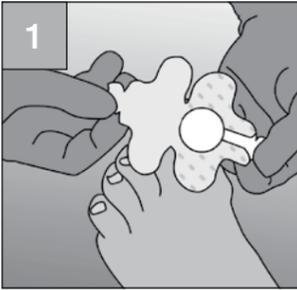

Hold the dressing by the side tab and remove the printed liner, exposing the adhesive border.

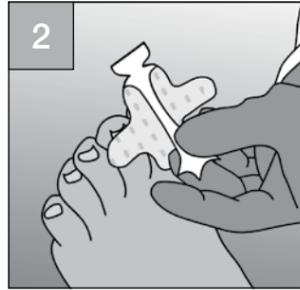

Grasp the paper tab on the top of the dressing and position the dressing over the wound.

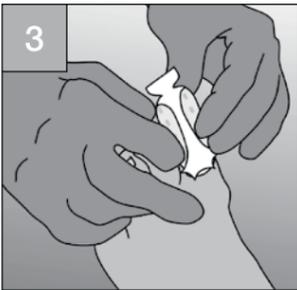

Gently press the adhesive border tabs to the skin, overlapping the tabs to conform to contours. Avoid stretching the adhesive tabs or skin during application.




Remove the paper tab.




Firmly press and conform the adhesive border to the skin.

Removal Technique — Mini Wrap

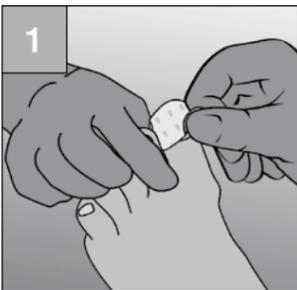


Gently lift the adhesive film border while pressing down on the skin.

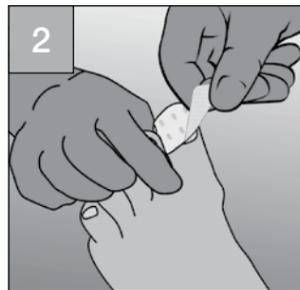


If there is difficulty lifting the film border, apply tape to the edge of the dressing and use the tape to lift.

Specialty Applications — Overlapping Dressings

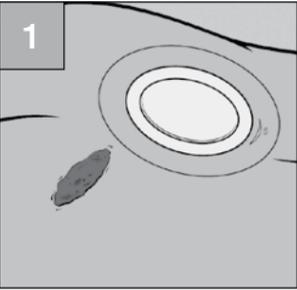

Select dressings to adequately cover wounds. Apply the first dressing per steps 1–4 in Dressing Application.

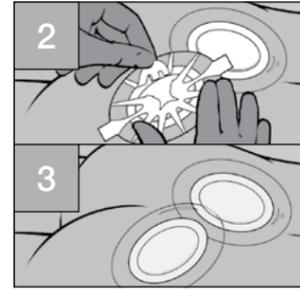

Overlap second dressing as needed to cover remaining wounds.

This method will avoid cutting.

Ordering Information: 3M™ Tegaderm™ High Performance Foam Adhesive Dressing

Product code	Overall dressing size	Absorbent pad size	Dressings/Box	Boxes/Case
90610 square	3½ in. x 3½ in. 8,8 cm x 8,8 cm	2 in. x 2 in. 5 cm x 5 cm	10	4
90611 small oval	4 in. x 4½ in. 10 cm x 11 cm	2½ in. x 3 in. 6 cm x 7,6 cm	10	4
90612 square	5½ in. x 5½ in. 14,3 cm x 14,3 cm	4 in. x 4 in. 10 cm x 10 cm	10	4
90613 medium oval	5½ in. x 6½ in. 14,3 cm x 15,6 cm	4 in. x 4½ in. 10 cm x 11 cm	5	6
90614 mini oval	2¾ in. x 3 in. 6,9 cm x 7,6 cm	1¼ in. x 1½ in. 3,1 cm x 3,8 cm	10	4
90616 large oval	7½ in. x 8¾ in. 19 cm x 22,2 cm	5½ in. x 6¾ in. 14 cm x 17,1 cm	5	3
90619 heel/elbow design	5½ in. x 5½ in. 13,97 cm x 13,97 cm	3 in. x 3 in. 7,62 cm x 7,62 cm	5	4
90615* mini wrap	2¾ in. x 2¾ in. 6,9 cm x 6,9 cm	1 in. round 2,5 cm x 2,5 cm	10	4

*The mini wrap dressing is constructed of a conformable, absorbent, polyurethane foam pad with a highly breathable, non-waterproof, film backing reinforced with soft cloth tape.

To learn more about 3M™ Tegaderm™ High Performance Foam Adhesive products, visit us at www.3M.com/TegFoam-App.

For more information about the Skin & Wound Care family of products, visit us at www.3M.com/Tegaderm, contact your 3M Skin & Wound Care representative or call the 3M Health Care Customer Helpline at **1-800-3M Helps (1-800-364-3577)**.



Considerations

- Observe the dressing frequently. As the dressing absorbs, exudate will wick to the top of the dressing and discoloration may be noticeable. When the exudate spreads to the edges of the dressing or the dressing begins to leak, a dressing change is indicated.
- 3M™ Tegaderm™ High Performance Foam Adhesive Dressing may be used as an absorbent cover dressing over wound fillers such as alginates or gauze packing.
- Dressing change frequency will depend on the type of wound, volume of exudate and clinical situation. Change at least every seven days or as indicated per treatment protocol.
- You may cut the dressing as needed to fit the size of the wound or accommodate wound contours. Seal the cut edge with 3M™ Tegaderm™ Transparent Dressing to provide waterproof barrier.

3M™ Tegaderm™ SIMPLE. DEPENDABLE. TRUSTED.
Dressings

3M

Skin & Wound Care Division
3M Health Care
3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
1 800 364-3577
www.3M.com/HealthCare

3M, Cavilon and Tegaderm are trademarks of 3M. Used under license in Canada.

Please recycle. Printed in Canada
© 2010, 3M. All rights reserved.
1005-01481-E

3M™ Tegaderm™ High Performance
Foam Adhesive Dressing



3M

Prior to Application:


Follow Universal Precautions and facility guidelines for infection control.

Wash or sanitize hands, set up a clean field with supplies, apply gloves.

Supplies You May Need:

- Gloves
- Saline or irrigation solution
- Gauze pads
- Clean towels or underpads to protect patient and surfaces
- 3M™ Tegaderm™ High Performance Foam Adhesive Dressing—select a dressing size to allow for 2.5 cm (1 inch) of foam to extend beyond the wound edge
- 3M™ Cavilon™ No Sting Barrier Film
- Appropriate protective wear:
 - Mask
 - Eyewear
 - Protective gown


Cleanse the Wound

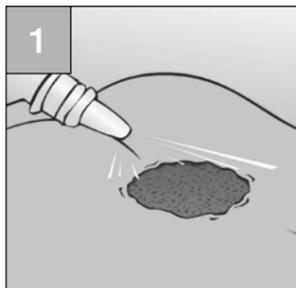

Cleanse wound and periwound skin according to your institution's protocol. Dry the area around the wound with dry gauze.


Note: Do not use the dressing in combination with oxidizing agents such as hydrogen peroxide or hypochlorite solution. These solutions may affect the structure and performance of the polyurethane foam.

Assess the Wound

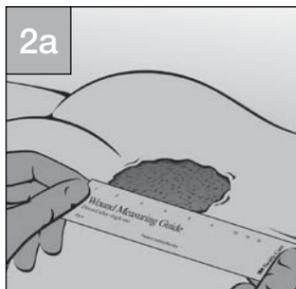


Observe the wound for signs of infection. If infection occurs, see a health care professional. 3M™ Tegaderm™ foam dressing may be used on infected wounds only under the care of a health care professional.

Remove soiled gloves and wash or sanitize hands. Apply clean gloves.

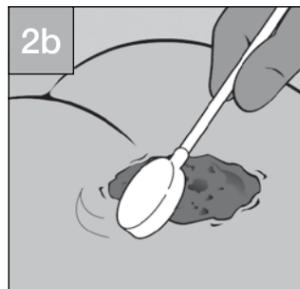


Note: If periwound skin is fragile or exposure to wound exudate is likely, apply a barrier film, such as 3M™ Cavilon™ No Sting Barrier Film. Let dry.


Dressing Application

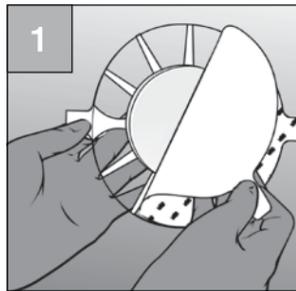


Hold the dressing by one of the side tabs and remove the printed liner to expose the adhesive border.

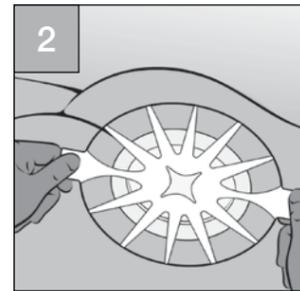

While holding the tabs, position the dressing over the wound and apply.

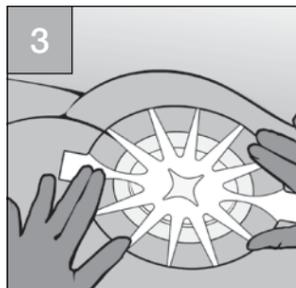


Gently press the adhesive film border to the skin. Note: Avoid stretching the dressing or the skin during application.

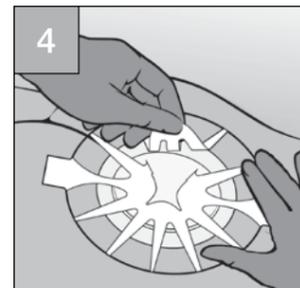


Beginning at the centre of the dressing, grasp one section of the paper carrier and pull toward the outer edge of the dressing while gently pressing down the film border. Repeat this for each of the remaining carrier sections until all paper carrier is removed.

continued from previous page

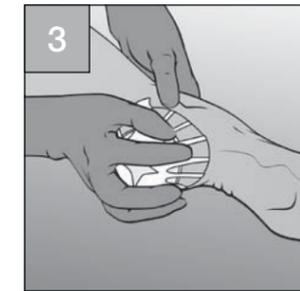


With one hand, press or "cup" the dressing onto the skin.

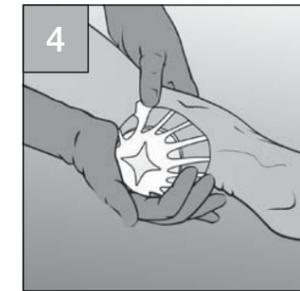


Continue to press and conform the dressing and adhesive border onto the skin.

Heel/Contour Surface Application (Catalog No. 90619 Heel/Elbow)

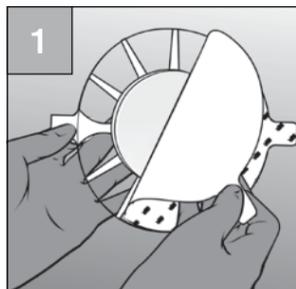


Hold the dressing by one of the side tabs and remove the printed liner, exposing the adhesive border.

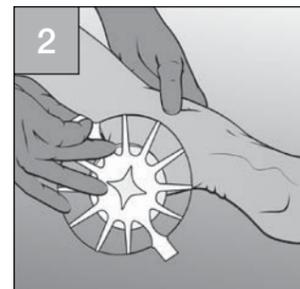


While stabilizing the leg or surrounding skin with one hand, position the dressing over the wound.

continued on next page

Removal Technique

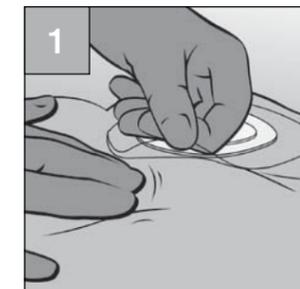

Gently lift the adhesive film border while pressing down on the skin.

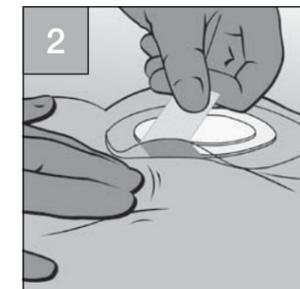

If there is difficulty lifting the film border, apply tape to the edge of the dressing and use the tape to lift.