

# Pressure Injury Prevention Protocol Implementation and Product Selection

Prevention	Deep Tissue Injury	Stage 1	Stage 2	Stage 3	Stage 4	Unstageable
		X	☹☹	☹☹☹	☹☹☹☹	X - ☹☹☹☹

## Suggested Nursing Interventions

	<b>Cleanse:</b> 3M™ Cavilon™ No-Rinse Skin Cleanser		<b>Cleanse:</b> 3M™ Cavilon™ No-Rinse Skin Cleanser		<b>Protect:</b> Peri-wound Tissue 3M™ Cavilon™ No Sting Barrier Film Spray or Wipe		<b>Protect:</b> Peri-wound Tissue 3M™ Cavilon™ No Sting Barrier Film Spray or Wipe	<b>Assess for healability. If not healable, do not debride – paint with povidone-iodine and keep dry.</b>	
	<b>Protect:</b> 3M™ Cavilon™ Durable Barrier Cream		<b>Protect:</b> 3M™ Cavilon™ Durable Barrier Cream or 3M™ Cavilon™ No Sting Barrier Film Spray or Wipe		<b>Cover:</b> 3M™ Tegaderm™ Absorbent Clear Acrylic Dressing		<b>Fill:</b> 3M™ Tegaderm™ Alginate Dressing in presence of moderate-to-heavy drainage to absorb exudate.		<b>If healable, consider:</b> 3M™ Nu-Gel™ Hydrogel with Alginate assists with natural autolytic debridement, while the alginate component enhances its absorptive capabilities.
	<b>Option to be considered:</b> 3M™ Tegaderm™ Silicone Foam Dressing with 7-day wear time		<b>Do not massage reddened area or bony prominences.</b>		<b>Option to be considered:</b> 3M™ Tegaderm™ Alginate Ag Silver Dressing		<b>If infected, consider:</b> 3M™ Tegaderm™ Alginate Ag Silver Dressing		<b>If infected, consider:</b> 3M™ Tegaderm™ Alginate Ag Silver Dressing
	<b>Option to be considered:</b> 3M™ Tegaderm™ Silicone Foam Dressing with 7-day wear time		<b>Option to be considered:</b> 3M™ Tegaderm™ Silicone Foam Dressing with 7-day wear time.		<b>Option to be considered:</b> 3M™ Tegaderm™ Silicone Foam Dressing 7-day wear time		<b>Option to be considered:</b> 3M™ Silvercel™ Dressing Family		<b>Option to be considered:</b> 3M™ Silvercel™ Dressing Family
	<b>Option to be considered:</b> 3M™ Cavilon™ Advanced Skin Protectant		<b>Use a soft silicone multi-layered foam dressing to help protect the skin for individuals at risk of pressure injuries.</b>		<b>Option to be considered:</b> 3M™ Inadine™ (PVP-I) Non Adherent Dressing		<b>Option to be considered:</b> 3M™ Inadine™ (PVP-I) Non Adherent Dressing		<b>Option to be considered:</b> 3M™ Inadine™ (PVP-I) Non Adherent Dressing
					<b>Cover:</b> 3M™ Tegaderm™ Silicone Foam Dressing with 7-day wear time		<b>Option to be considered:</b> 3M™ Tegaderm™ Silicone Foam Dressing with 7-day wear time		<b>Cover:</b> 3M™ Tegaderm™ Silicone Foam with 7-day wear time.
					<b>Option to be considered:</b> 3M™ Tegaderm™ High Performance Foam Adhesive Dressing		<b>Option to be considered:</b> 3M™ Tegaderm™ High Performance Foam Adhesive Dressing		<b>Option to be considered:</b> 3M™ Tegaderm™ High Performance Foam Adhesive Dressing
					<b>Option to be considered:</b> 3M™ Snap™ Therapy System		<b>Option to be considered:</b> 3M™ Snap™ Therapy System		<b>Option to be considered:</b> 3M™ Snap™ Therapy System
					<b>Option to be considered:</b> 3M™ V.A.C.® Therapy System		<b>Option to be considered:</b> 3M™ V.A.C.® Therapy System		<b>Option to be considered:</b> 3M™ V.A.C.® Therapy System

**DISCLAIMER:** This material is provided as general information for the health care professional regarding skin and wound care assessment and various treatments. This material contains statements, technical information and guidance, which are based on current literature and clinical information at the time the material was produced, and which we believe was reliable when created. However, assessment and treatment practices continually change, such that the completeness or accuracy of its content cannot be guaranteed over time. 3M recommends that the information herein be used as a guide only. In all cases, professional clinical judgment must be used for assessment, intervention, and evaluation in each clinical situation. 3M shall not be liable or responsible for the consequences of any actions taken on the basis of the guidelines contained in, or errors or omissions from, this information. This material may not be copied or used in part or in full without the express permission from 3M.

Source: European Pressure Ulcer Advisory Panel, National Pressure Injury Advisory Panel and Pan Pacific Pressure Injury Alliance. Prevention and Treatment of Pressure Ulcers/Injuries: Quick Reference Guide. Emily Haesler (Ed.). EPUAP/NPIAP/PPPIA: 2019.

3M, 3M Science. Applied to Life., Cavilon, Tegaderm, Adaptic, Silvercel, Inadine, Snap, Nu-Gel, and V.A.C. are trademarks of 3M. Used under license in Canada. All other trademarks are the property of their respective owners. © 2022, 3M. All rights reserved. 2206-24264-E CA\_EN\_70-2013-1460-9

Available in Canada from your authorized 3M-KCI distributors.  
KCI USA, Inc., a 3M Company  
KCI owned and operated by 3M Company

<b>3M Medical Solutions Division</b> 3M Canada Company 300 Tartan Drive London, ON N5V 4M9 1-800-563-2921 3M.ca/Medical	<b>3M Company</b> 2510 Conway Avenue St. Paul, MN 55144 USA 1-800-228-3957 3M.com	<b>KCI Medical Canada Inc.</b> 75 Courtneypark Dr W, Unit 4 Mississauga, ON L5W 0E3	<b>KCI USA, INC.</b> 12930 IH 10 West San Antonio, TX 78249
--	--	---	--