



3M™ Cavilon™ No Sting Barrier Film



Indications:

- Damaged skin: excoriated, reddened or denuded
- Barrier to protect peri-wound area
- Skin tears
- Small cuts and scrapes
- Skin protector for sensitive and fragile skin
- Skin protector for frequent dressing changes

Directions for use:

1. Skin should be clean and dry prior to application of Cavilon™ No Sting Barrier Film.
- 2 a) Wipe or foam wand: Apply a uniform coating of film over entire area of concern.
b) Spray bottle: Hold nozzle 10 cm to 15 cm (4" to 6") from skin and apply a smooth coating over the entire area of concern while moving spray in a sweeping motion.
3. If an area is missed, reapply to that area only after the first application of Cavilon™ No Sting Barrier Film has dried (approx. 30 sec).
4. If Cavilon™ No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, ensure that these areas of skin are separated to allow coating to dry thoroughly before allowing skin to return to normal position.

Continued on back...



Prevention that Rises Above

3M

3M™ Cavilon™ No Sting Barrier Film

Directions for use cont'd:

5. Reapplication is recommended at least every 24-72 hours depending on cleansing frequency.
6. Reapplication is required after every adhesive dressing change.
7. Removal of the film is not necessary before every application. If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the area, and reapply Cavilon™ No Sting Barrier Film following the directions above.
8. Replace the cap over the spray nozzle when not in use.

This product may be used on adults, children and infants over one month of age.



Ordering Information:

Cat. No.	Size	Items/Box	Boxes/Case
3343E	1.0 mL peel-open wand	25	4
3344E	1.0 mL wipe	30	6
3345E	3.0 mL wand	25	4
3346E	28.0 mL spray	12	1



For more information, contact your 3M Critical & Chronic Care Solutions Representative or call the 3M Health Care Customer Helpline at 1-800-364-3577.

Available in Canada from:
3M Critical & Chronic Care Solutions
3M Canada Company
P.O. Box 5757
London, Ontario N6A 4T1, Canada
1-800-364-3577
www.3M.ca/skinwound

3M Health Care
2510 Conway Avenue
St. Paul, MN 55144
USA
1-800-228-3957
www.3M.com/Cavilon