

3M Science.
Applied to Life.™

3M Oral Care

**3M post and core solution.
Complexity simplified.**

3M post and core solution. Featuring 3M™ RelyX™ Fiber Post 3D.

For predictable results – in less time.

We know: The endodontic procedure is very complex. So the simpler and more reliable the endodontic restorative procedure is, the better – for both the dentist and the patient. The excellent 3M post and core solution is a big step forward in this regard: With significantly fewer clinical steps and fewer products, it lets you work faster, easier and more efficiently.

All you need to do is take four steps.

The 3M post and core solution not only simplifies your work but also helps you deliver predictable, robust outcomes – from root to core. All you need is the 3M™ RelyX™ Fiber Post 3D Glass Fiber Post plus three other easy-to-use products from 3M Oral Care: 3M™ RelyX™ Unicem 2 Automix Cement, 3M™ Scotchbond™ Universal Adhesive, and 3M™ Filtek™ One Bulk Fill Restorative.

These four products – each of them proven to make your work more reliable and efficient – have one common objective: Simplify the complex!

Fewer clinical steps, fewer products.




Post

3M™ RelyX™ Fiber Post 3D Glass Fiber Post – With a unique 3D design for more confidence

RelyX Fiber Post 3D builds on the 8 years of clinical success of the proven product RelyX Fiber Post. Now, we have further improved it to enhance your peace of mind:

- High mechanical retention to the core build-up material thanks to coronal 3D macro retention.
- Strong bond in the root canal due to its microporous surface without a silanating step.
- Better post position control thanks to excellent radiopacity.
- Low risk of root fracture due to dentin-like elasticity.
- More esthetic option than a metal post.



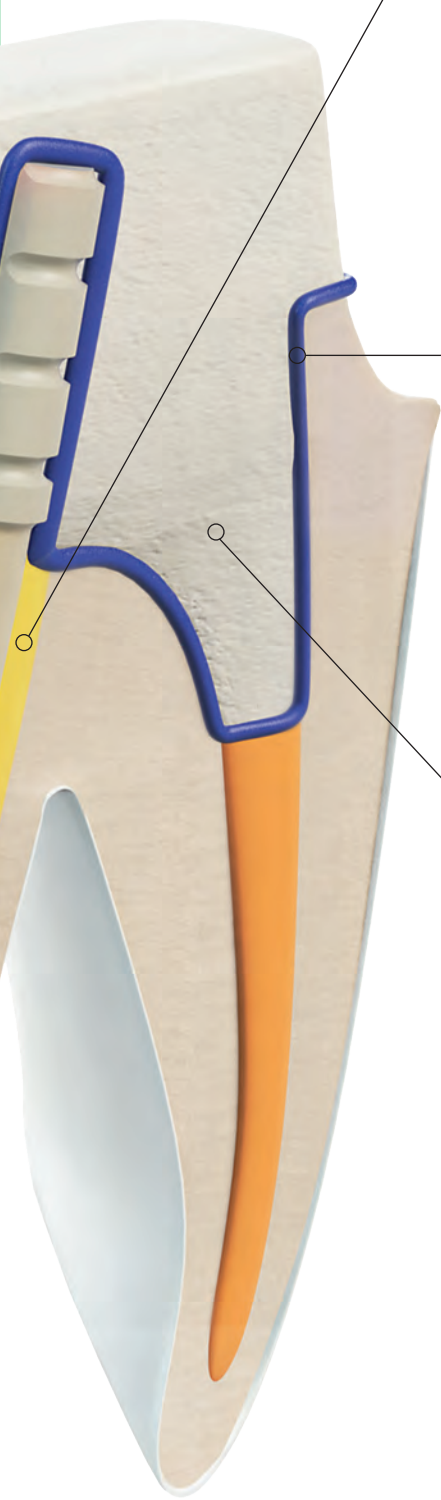

Cement



3M™ RelyX™ Unicem 2 Automix Self-Adhesive Universal Resin Cement

RelyX Unicem 2 Automix Cement not only offers excellent bond strength but also saves you precious time, as you do not need to etch, prime or bond.

The handy automix syringe, together with the endo tip, lets you apply the ready-mixed cement directly in the root canal from bottom to top.

With one application – and virtually no voids.

Adhesive



3M™ Scotchbond™ Universal Adhesive

Scotchbond Universal Adhesive provides a reliable bond of the core build-up material to the post and tooth structure – with no need for an additional primer.

One application – done!

Core build-up



3M™ Filtek™ One Bulk Fill Restorative

Excellent material and handling characteristics make Filtek One Bulk Fill Restorative a great choice of core build-up material. With one-step increments up to 5mm, core build-ups can be done faster and easier.

3M™ RelyX™ Fiber Post 3D Glass Fiber Posts are available in four different sizes. Color codes ensure an accurate match to the drills.



Five drill sizes (including one universal drill).



Ordering Information

| Item No. | Product Information |
|---|--|
| 3M™ RelyX™ Fiber Post 3D Glass Fiber Post | |
| 56957 | Introductory Kit 20 posts sizes 0, 1, 2, 3 (5 posts of each size), 5 drills sizes 0, 1, 2, 3, Universal (1 drill of each size) |
| 56958 | Trial Kit with 3M™ RelyX™ Unicem 2 Automix Cement 10 posts sizes 1, 2 (5 posts of each size), 3 drills sizes 1, 2, Universal (1 drill of each size), 1 – 3M™ RelyX™ Unicem 2 Automix Syringe A2 – 8.5g, 10 mixing tips, 10 endo tips |
| 56960 | Post and Core Build-up Kit with 3M™ RelyX™ Unicem 2 Automix Cement 1 Trial Kit 3M™ RelyX™ Fiber Post 3D / 3M™ RelyX™ Unicem 2 Automix Cement, 20 capsules 3M™ Filtek™ One Bulk Fill Restorative A3 – 0.2g each, 1 vial 3M™ Scotchbond™ Universal Adhesive – 5ml |
| Refills posts and drills | |
| 56948 | 10 posts, size 0 (1.1 mm dia), white |
| 56949 | 10 posts, size 1 (1.3 mm dia), yellow |
| 56950 | 10 posts, size 2 (1.6 mm dia), red |
| 56951 | 10 posts, size 3 (1.9 mm dia), blue |
| 56873 | 1 drill, size 0 (1.1 mm dia), white |
| 56864 | 1 drill, size 1 (1.3 mm dia), yellow |
| 56865 | 1 drill, size 2 (1.6 mm dia), red |
| 56866 | 1 drill, size 3 (1.9 mm dia), blue |
| 56867 | 1 Universal drill |
| 3M™ RelyX™ Unicem 2 Automix Self-Adhesive Resin Cement | |
| Refills | |
| 56846 | Shade A2 Universal 1 syringe – 8.5g, 15 mixing tips (regular), 5 mixing tips (wide), 5 endo tips |
| 56847 | Shade A3 Opaque 1 syringe – 8.5g, 15 mixing tips (regular), 5 mixing tips (wide), 5 endo tips |
| 56848 | Shade Translucent 1 syringe – 8.5g, 15 mixing tips (regular), 5 mixing tips (wide), 5 endo tips |
| Value Packs | |
| 56849 | Shade A2 Universal 3 syringes – 8.5g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips |
| 56857 | Shade A3 Opaque 3 syringes – 8.5g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips |
| 56858 | Shade Translucent 3 syringes – 8.5g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips |
| Accessories – Cement Mixing Tips | |
| 56851 | 30 mixing tips (regular) |
| 56919 | 15 mixing tips (wide) |
| 56918 | 15 mixing tips (wide), 15 intra-oral tips |

| Item No. | Product Information | |
|---|--|---|
| 3M™ Scotchbond™ Universal Adhesive | | |
| 41254 | Introductory Kit 1 vial 3M™ Scotchbond™ Universal Adhesive – 5ml, 50 disposable mixing wells, 50 disposable microbrushes, 1 etchant syringe – 3ml, 25 dispensing tips for etchant syringe | |
| 41258 | Refill 1 vial 3M™ Scotchbond™ Universal Adhesive – 5ml | |
| 3M™ Filtek™ One Bulk Fill Restorative | | |
| Refill | Item No. | Item No. |
| Shade | Syringe incl. 1 syringe – 4g | Capsule incl. 20 capsules – 0.2g each |
| A1 | 4866A1 | 4867A1 |
| A2 | 4866A2 | 4867A2 |
| A3 | 4866A3 | 4867A3 |
| B1 | 4866B1 | 4867B1 |
| C2 | 4866C2 | 4867C2 |
| 3M™ Filtek™ Bulk Fill Flowable Restorative | | |
| Refill | Item No. | Item No. |
| Shade | Syringe incl. 2 syringes – 2g each, 20 dispensing tips | Capsule incl. 15 capsules – 0.2g each |
| A1 | 4862A1 | 4861A1 |
| A2 | 4862A2 | 4861A2 |
| A3 | 4862A3 | 4861A3 |
| Universal | 4862U | 4861U |
| 3740T | 20 dispensing tips 19-gauge | |
| 3M™ Retraction Capsule | | |
| 56941 | Refill Pack: 25 capsules | |
| 56942 | Value Pack: 50 capsules | |
| 3M™ Elipar™ DeepCure-S LED Curing Light | | |
| 76975 | 1 stainless steel handpiece (cordless), charging base (230V), Li-ion battery, 10 mm light guide, eye shield | |
| 3M™ Elipar™ DeepCure-L LED Curing Light | | |
| 76973 | 1 lightweight handpiece (built-in Li-ion battery), universal power supply with 5 adapters, 10 mm light guide, eye shield, 3 curing discs | |



3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000
USA
1-800-634-2249

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

3M.com/RelyX3D

3M, Elipar, Filtek, RelyX and Scotchbond are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada. All other trademarks are owned by other companies. Please recycle. Printed in U.S.A. © 3M 2017. All rights reserved. 70-2013-0728-0 (Rev 9/2017)