Competitive Product and Procedure Comparison

3M™ RelyX™ Fiber Post 3D Glass Fiber Post

Offered by 3M



Summary - Advantages of 3M Solution over DMG

- Faster and simpler 3M post and core solution with proven products (Fig. 1)
- Reliable post and core build-up procedure with fewer clinical steps and fewer products (Fig. 1)
- Easier cementation with 3M™ RelyX™ Unicem 2 Automix Cement: no post silanating, no root canal conditioning, no application of cement to post, no lentulo usage (endo tip application of RelyX Unicem 2 Automix enables a virtually void-free cementation (Fig 1.))
- Easier bonding with 3M™ Scotchbond™ Universal Adhesive for the core build-up: No etching or priming (Fig. 1)

3M post and core solution compared to DMG

	Post cementation							Core build-up				
	Apply etching gel	Apply primer	Mix adhesive	Apply adhesive	Condition post	Apply cement	Place post and cure	Apply etching gel	Apply primer	Mix adhesive	Apply adhesive	Core build-up
3М	-	-	-	-	-		Compt Compt	-	-	-	alt	T
DMG		-			-	Th						

Fig. 1: Less components and steps compared to LuxaPost & LuxaCore System. Source: Steps according to instructions for use of LuxaPost, LuxaCore and 3M™ RelyX™ Fiber Post 3D.

RelyX Fiber Post 3D - Major features

- No pre-treatment of the post needed due to the micro-porosity at the surface (Fig. 2)
- Integrated 3D macro retention in the coronal part for a high bond strength to the core build-up (Fig. 3)
- Excellent radiopacity for better post position control (Fig. 4)

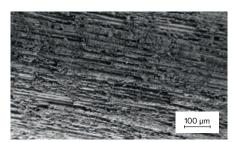


Fig. 2: Surface (SEM, 200x magnification) of 3M" RelyX" Fiber Post 3D.

Source: R. Peez, R. Hampe, S. Hader, E. Popp,

J. Edgington: Evaluation of a New Cement Post and Core Procedure System, J Dent Res 94 (Spec Iss A): 0105, 2015, (www.dentalresearch.org).



Fig. 3: The coronal macro retention of 3M™ RelyX™ Fiber Post 3D give a high and secure mechanical retention to the core build-up material, immediate depth control and auidance for post shortening.

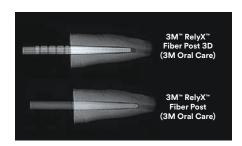


Fig. 4: Comparative radiography of both posts. Source: THE DENTAL ADVISOR, Research Report Number 70 – June 2015.



LuxaPost

Glass Fiber Post

Offered by DMG*



Product description and benefits:

(As stated by the manufacturer)

Glass fiber reinforced composite post with uniquely designed coronal macro-retention. Embedded in a fast and simple 3M post and core solution for a secure bond from root to core.

Glass fiber reinforced composite post with a similar flexural modulus to LuxaCore Z and natural dentin.

Cement



3M™ RelyX™ Unicem 2 Automix Cement

- Self-adhesive universal resin cement, no need for etching, priming and bonding
- Reduces significant number of steps
- Endo tip allows easy and virtually void-free application
- Delivers reliable and strong bond of post to root canal

LuxaCore Z

- Excellent flow properties
- With zirconium dioxide and nanotechnology
- Excellent flow properties which guarantee optimum adaptation to the shape of cavity walls and posts
- \bullet Film thickness of just 20 μm

Post



3M™ RelyX™ Fiber Post 3D

- High mechanical retention to core due to coronal macro retention
- Secure bond in the root canal due to superficial micro-porosity
- High radiopacity
- Dentin-like elasticity to avoid root fracture
- Esthetic alternative to metal posts
- The flattened coronal macro retention offer a safe grip with the tweezer

LuxaPost

- 2 tiny coronal retention grooves
- Pre-silanzised
- Radiopaque
- Dentin-like flexural modulus: reduced risk of root fracture

Adhesive



3M™ Scotchbond™ Universal Adhesive

- Light-curing, self-adhesive
- Reliable and strong bond to root and core
- Bonds to all surfaces including enamel, dentin, glass ceramic, zirconia, noble and non-precious alloys, and composites – without additional primer
- One application of adhesive sufficient (no etching or priming)
- Combined total-etch, self-etch and selective-etch adhesive

LuxaBond-Total Etch

(LuxaBond Pre-Bond & LuxaBond Adhesive A+B)

- Dual-curing (Hardens even without light)
- No premature polymerization of bonding »pools«
- Compatible with self-curing and dual-curing materials:
 Suitable for all applications with LuxaCore
- Pre-Bond facilitates quicker curing of the bond mix in the dentin for superior retention
- Easy root canal application with special endo brushes
- Does not contain acetone

Core build-up



3M™ Filtek™ One Bulk Fill Restorative

- Improved opacity and esthetics
- Excellent cavity adaptation
- One-step placement up to 5 mm
- Excellent handling and sculptability
- Excellent wear resistance and polish retention
- High radiopacity

LuxaCore Z

- Excellent flow properties
- Extremely high compressive strength
- Cuts like dentin

How it works

- Clean RelyX Fiber Post 3D with alcohol
- Apply RelyX Unicem 2 Automix Cement into root canal
- Insert post into root canal
- Apply Scotchbond Universal Adhesive to the coronal part of post and tooth
- Build-up core with Filtek One Bulk Fill Restorative
- Clean post with alcohol
- Apply LuxaBond Pre-Bond into root canal
- Mix LuxaBond Adhesive A and LuxaBond Adhesive B
- Apply the mixture of LuxaBond A and LuxaBond B into root canal
- Apply cement into root canal
- Insert post into root canal
- Mix LuxaBond Adhesive A and LuxaBond Adhesive B
- Apply the mixture of LuxaBond to post and tooth
- Build-up core with LuxaCore Z



*Information obtained from http://www.dmg-dental.com/products/

3M Oral Care

2510 Conway Avenue St. Paul, MN 55144-1000 USA 1-800-634-2249

3M Canada

Post Office Box 5757 London, Ontario N6A 4T1 Canada 1-888-363-3685

3M.com/dental

3M, ESPE, Filtek, RelyX and Scotchbond are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada. All other trademarks are owned by other companies. Please recycle. Printed in U.S.A. © 3M 2017. All rights reserved. 70-2013-0732-2 (Rev. 11/2017)