

3M Science.
Applied to Life.™

Capture precise detail.

3M™ ESPE™ Intra-oral Syringe


Restorations with accurate margins drop into place. Now you can apply wash material exactly where it's needed to capture every marginal detail. Patient-friendly and ergonomic, these little syringes make dispensing more efficient, with up to 67% less waste than hand dispensers.




Prefill up to
- 12 hrs. -
in advance




Steps for using the 3M™ ESPE™ Intra-oral Syringe

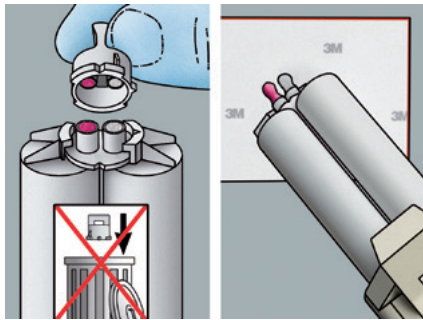


Remove the cartridge cap. Bleed a small amount of material from the cartridge onto mixing pad and discard.

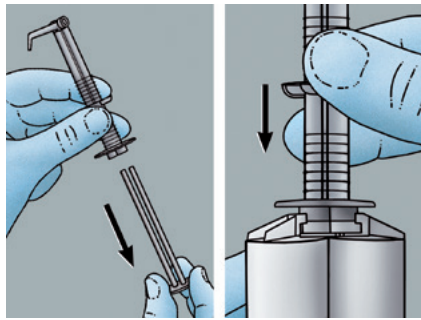


Remove the plunger of the intra-oral syringe. Place on the openings of Garant™ cartridge and attach.

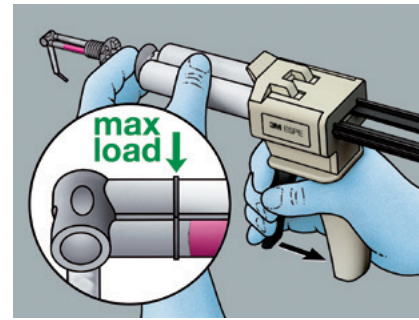


Hold firmly in place while filling. Leave mixing tip in angled position; do not fold open. Do not fill syringe past the maximum fill line.

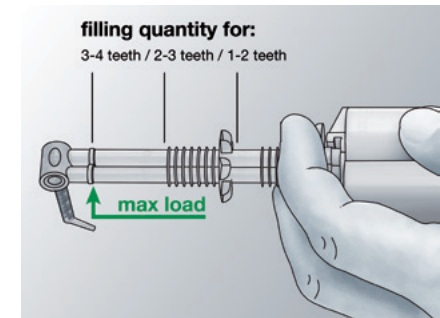


The content of an Intra-oral Syringe is sufficient for syringing about 2 – 4 preparations, depending on the quantity applied around each unit.

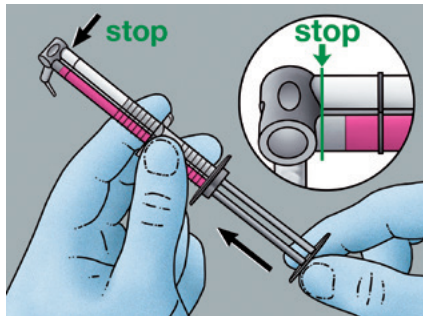


Replace plunger in the syringe, pushing paste no further than to the joint of the mixing tip.

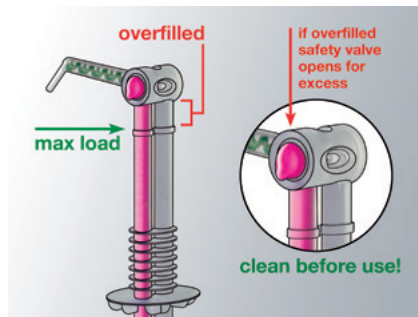

Any paste extruded on the sides of the mixing tip joint must be wiped off before the application in the mouth.

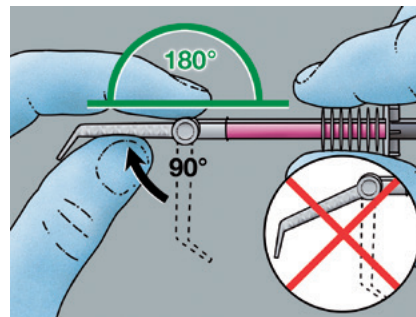

Immediately before application, fold out the mixing tip of the syringe at a 180° angle (until resistance is felt) to align with syringe. Do not bend the tip back down afterwards.

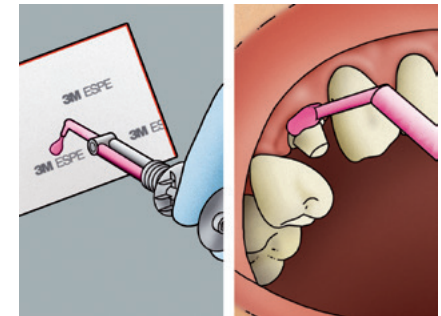

Press out a small quantity of paste onto a mixing pad until the base paste and catalyst flow out in an even, homogeneous mixture. Immediately syringe material around prepared tooth.



www.3M.com/GreatImpressions

3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
Phone 1-888-363-3685

NOTE: Intra-oral syringes cannot be autoclaved and must only be used on one patient! Refilling a syringe is prohibited.

3M, 3M Science. Applied to Life. and ESPE are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2016. All rights reserved. Garant is not a trademark of 3M.