

3M Science.
Applied to Life.™



Retract tissue gently.

3M™ Astringent Retraction Paste


A great impression starts with controlling tissue and moisture. Our capsule's extra-fine tip delivers hemostatic paste where it belongs—right into the sulcus—to gently retract tissue and control bleeding.




Delivers a 15% aluminum chloride astringent paste into the sulcus for gentle retraction of gingival tissue.




Steps for using 3M™ Astringent Retraction Paste

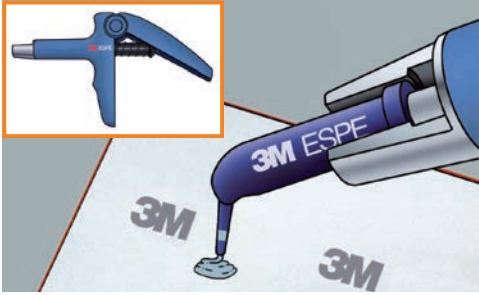


Place capsule firmly in dispenser. Remove cap immediately before use and discard. Dispense small quantity of paste onto a mixing pad and discard. NOTE: Rounded capsule tip must not be shortened because this would result in sharp edges.

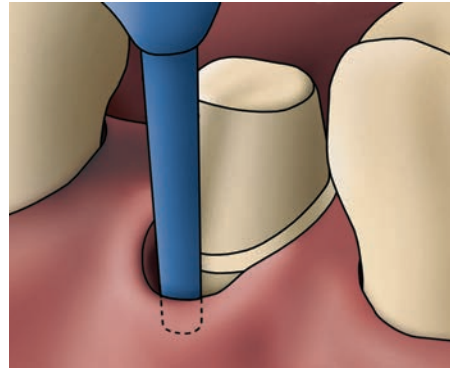

Rinse sulcus thoroughly with water and dry lightly with air before applying the paste. Insert capsule tip into the sulcus.

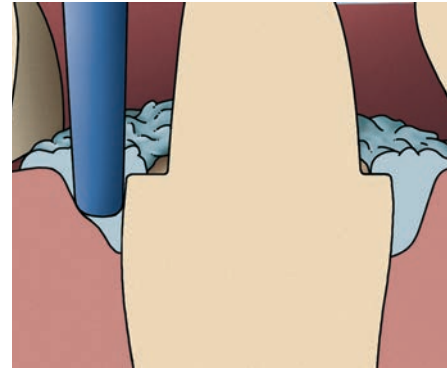


Slowly and steadily inject astringent retraction paste into sulcus. Completely fill the sulcus.

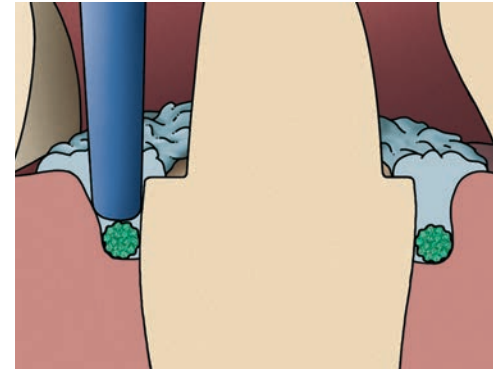

If required by the clinical situation, the retraction paste can be used in combination with a cord to open the sulcus further.

The retraction paste can be used with or without cord.

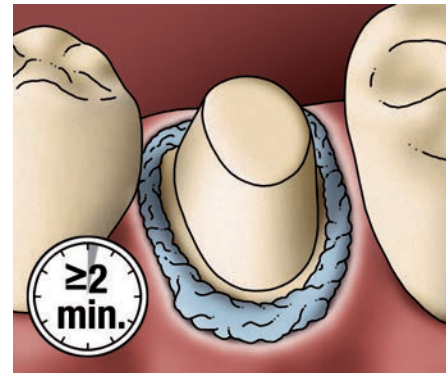

Allow the astringent retraction paste to react for at least 2 minutes in the sulcus. Keep all moisture away from the site during this period. NOTE: An overly long residence time may cause damage to tissue.



Remove the retraction paste completely from the sulcus and surrounding areas with air, water and suction. NOTE: The patient must not swallow the retraction paste.

3M

3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
Phone 1-888-363-3685

3M.com/GreatImpressions

3M is a trademark of 3M or 3M Deutschland GmbH. Used under license in Canada.
© 3M 2018. All rights reserved. (Rev. 2018)