

Live Link:



753378

FUTURO™ Precision Fit Ankle Support

Details

Provides compression and support for stiff, weak or injured ankles. It also helps provide protection from further injury.

Benefits

- Low profile, contoured shape is designed for optimal comfort and support
- Odor resistant
- Flexible comfort strap adjusts for a personalized fit and support
- Easy to put on and take off
- Provides enhanced breathability compared to neoprene
- Fits easily and comfortably inside your shoe
- Enhanced heel strap for added comfort

Care Instructions

Close fastener prior to washing. Hand wash with mild soap in warm (not hot) water. Rinse well and air dry.

DO NOT: machine wash, tumble dry, dry clean, or use bleach or fabric softeners.

How to Wear



Step 1

Position support with smooth side against skin. Place heel securely in loop. Pull support to side of foot and wrap over the top of foot toward inside of ankle.



Step 2

Ensure that the woven area is at the front of the ankle.



Step 3

Wrap support around back of ankle, then toward front of ankle. Attach hook fastener to support. Re-wrap support with varying degrees of tension until desired support and comfort are achieved.