

Live Link:



610664



FUTURO™ Soft Cervical Collar

Details

Designed to provide relief from pinched nerves or tightness in neck.

Benefits


- Adjustable straps for both height and circumference
- Chin strap adjusts collar height for comfort and fit
- Breathable foam covered with soft, smooth knit material
- Easy to put on and take off

Care Instructions

Close fasteners prior to washing. Hand wash with mild soap in warm (not hot) water. Rinse well and air dry.

DO NOT: machine wash, tumble dry, dry clean, or use bleach or fabric softeners.

How to Wear

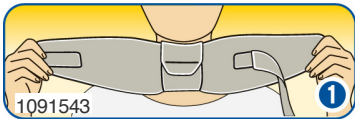

Step 1

To apply, hold in both hands with the flap of the center chin strap facing away from you and the hook fastener in the right hand, facing the "outside" of the collar. Place the collar at the front of the neck, chin strap centered under the chin.




Step 2

Wrap the left end of the collar around the back of the neck, and attach to opposite end.



Step 3

Then extend the soft pile strip from the left side to overlap and firmly engage the hook fastener strip. Adjust fasteners until fit is firm, yet comfortable.

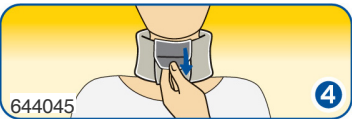

Step 4

Holding the collar firmly in place, disengage the chin strap flap. Tighten or loosen the chin strap until the collar fits comfortably but with firm support under the chin. Re-engage the fastener.