

3M

Curoso™

Disinfecting Port Protectors

HOW TO USE

Curoso Jet™

Disinfecting Cap for Needleless Connectors

Disinfects in 1 minute — protects ports for up to 7 days.



Peel
the foil from the 3M™ Curoso™ Jet
Disinfecting Cap.



Push & Twist
the Curoso Jet disinfecting cap
clockwise onto the end of the
needleless connector.

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab prior to using cap.
- Do not use cap if seal is broken, torn, or punctured.
- Cap must remain on needleless connector for minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the Curoso Jet disinfecting cap that has been in place for a minimum of one (1) minute.
- Always place a new Curoso Jet disinfecting cap on needleless connector following completion of a valve access.
- Dispose of the Curoso Jet disinfecting cap after every use.
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses.

WARNING: To avoid potential for injury - use ONLY on needleless connectors.

CAUTION: Potential choking hazard.



Green means clean.

3M

Critical & Chronic Care Solutions Division
3M Health Care
2510 Conway Avenue
St. Paul, MN 55144 USA

1 800 228-3957
www.3M.com/C3SD

Please recycle this poster. Printed in U.S.A.
© 3M 2016. All rights reserved.
3M, Curoso and Curoso Jet are trademarks of 3M.
70-2011-5725-5

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA).
Single Use Only. Rx Only. This device is not
made with natural rubber latex or plasticiser
Diethylhexylphthalate (DEHP). Refer to current
device labeling for indications for use, warnings
and product details.