



# Curoso™

Disinfecting Port Protectors

## HOW TO USE

# Curoso™

Disinfecting Cap for Needleless Connectors

**Disinfects in 1 minute — protects ports for up to 7 days.**



**Peel**  
the foil from the 3M™ Curoso™  
Disinfecting Cap.



**Push & Twist**  
the Curoso disinfecting cap clockwise  
onto the end of the needleless connector.

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for a minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the Curoso disinfecting cap that has been in place for a minimum of one (1) minute.
- Always place a new Curoso disinfecting cap on needleless connector after each use.
- Dispose of the Curoso disinfecting cap after every use.
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses.

**WARNING: To avoid potential for injury, use only on needleless connectors.**

**CAUTION: Potential choking hazard.**



**Green means clean.**



Critical & Chronic Care Solutions Division  
3M Health Care  
2510 Conway Avenue  
St. Paul, MN 55144 USA

1 800 228-3957  
www.3M.com/C3SD

Please recycle this poster. Printed in U.S.A.  
© 3M 2016. All rights reserved.  
3M and Curoso are trademarks of 3M.  
70-2011-5738-8

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA).  
Single Use Only. Rx Only. This device is not  
made with natural rubber latex or plasticiser  
Diethylhexylphthalate (DEHP). Refer to current  
device labeling for indications for use, warnings  
and product details.