



CuroS™

Disinfecting Port Protectors

HOW TO USE

CuroS Tips™

Disinfecting Cap for Male Luers

Disinfects in 1 minute — protects male luers for up to 7 days.



Peel
3M™ CuroS Tips™ disinfecting cap from foil strip.



Push & Twist
the CuroS Tips disinfecting cap clockwise onto the end of the male luer.

- If male luer connection on I.V. line is heavily soiled, clean with 70% isopropyl alcohol swab before using CuroS Tips disinfecting cap.
- Do not use if foil is broken, torn or punctured.
- CuroS Tips disinfecting cap must remain on male luer connection for a minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the CuroS Tips disinfecting cap that has been in place for a minimum of one (1) minute.
- Always place a new CuroS Tips disinfecting cap on male luer connection after each use.
- Dispose of the CuroS Tips disinfecting cap after every use; do not reuse.

CAUTION: Potential choking hazard.

 **Green means clean.**



Critical & Chronic Care Solutions Division
3M Health Care
2510 Conway Avenue
St. Paul, MN 55144 USA

1 800 228-3957
www.3M.com/C3SD

Please recycle this poster. Printed in U.S.A.
© 3M 2016. All rights reserved.
3M, CuroS and CuroS Tips are trademarks of 3M.
70-2011-5798-2

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA).
Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.