

3M

Curos™

Disinfecting Port Protectors

HOW TO USE: DISINFECTING CAP FOR HOME INFUSION

Curos™


Disinfecting Cap for Needleless Connectors

Disinfects in 1 minute — protects ports for up to 7 days.



Peel

the foil from the 3M™ Curos™ Disinfecting Cap

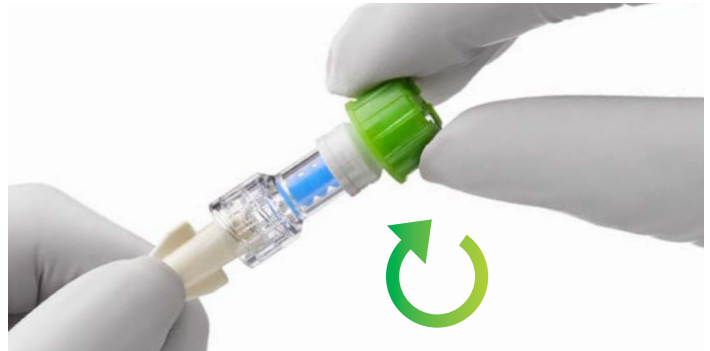

Push & Twist

the Curos disinfecting cap clockwise onto the end of the needleless connector.

- If the valve becomes soiled with blood or other contaminants, it should be swabbed with a 70% isopropyl alcohol pad for 15 seconds before placing a new Curos cap on the clean valve.
- As a precaution, replace the Curos cap if it gets wet.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for a minimum of one (1) minute and may remain on for seven (7) days if not removed.
- Always place a new Curos disinfecting cap on needleless connector following completion of a valve access.
- Dispose of the Curos disinfecting cap after every use.

WARNING: To avoid potential for injury, use only on needleless connectors.

CAUTION: Potential choking hazard.



Green means clean.

3M

Critical & Chronic Care Solutions Division
3M Health Care
2510 Conway Avenue
St. Paul, MN 55144 USA

1 800 228-3957
www.3M.com/C3SD

3M Canada
P.O. Box 5757
London, Ontario N6A 4T1
Canada

1 800 364-3577
www.3M.ca/HealthCare

Please recycle. Printed in U.S.A.
© 3M 2016. All rights reserved.
3M and Curos are trademarks of 3M.
70-2011-5736-2

STERILE	R
---------	---

Active Ingredient: 70% isopropyl alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.