



3M™ Tegaderm™ I.V. Advanced Securement
Dressings 1680 & 1682

Application and Removal Guide

3M™ Tegaderm™ I.V. Advanced Securement Dressings

Application & Removal Guide — 1680 & 1682



1. Insert I.V. catheter according to facility protocol.



2. Hold dressing with one hand; grasp the non-adhesive portion of the dressing. Orient notch ends of dressing (the fingers of the dressing) toward the patient's fingers.



3. Position window portion of dressing over insertion site with the top of the keyhole notch lined up at the connection between the I.V. catheter hub and I.V. tubing.



4. Apply dressing over catheter hub and use gentle but firm pressure around window border and outward toward dressing edges to secure. Gently lift tubing connection area and press border firmly underneath catheter hub.



5. Slowly peel frame away from border edge while simultaneously applying firm pressure to border edge. Set paper frame aside—do not discard.



6. Smooth dressing from center toward edges using firm pressure to ensure optimal adhesion.



7. Using largest securement strip (found on paper frame), apply strip half firmly onto dressing at the tubing/catheter hub connection site and over to other side.



8. Use second securement strip under the tubing/catheter hub connection site. Press both strips firmly into skin.



9. Loop connector tubing into place and secure with third securement strip (documentation label).

Removal



Remove securement strips away from dressing.



Gently grasp dressing edge below the keyhole notch.



Ensure the I.V. is stabilised as dressing is lifted from skin using low and slow technique.



Peel dressing toward catheter insertion site.



3M Critical & Chronic Care Solutions Division

3M Australia Pty Limited
Building A, 1 Rivett Road
North Ryde
NSW 2113
1300 363 878
www.3M.com.au/healthcare

3M New Zealand Limited
94 Apollo Drive
Rosedale,
Auckland 0632
0800 80 81 82
www.3M.co.nz

3M and Tegaderm are trademarks of 3M Company.
Please recycle. Printed in Australia. ©3M 2016.
All rights reserved. PB7778/0416_1680_1682