

Application Steps

3M™ Bair Hugger™ Multi-position upper body warming blanket - Model 62200



1. Place

Once surgical positioning is complete, place the blanket on the patient. The small holes should be in contact with the patient's skin.



4. Adhere

Adhere to patient in the desired location (adhesive strip toward surgical site).



7. Connect

Connect hose to blanket using desired port (not over groin area).



Final Beach Chair Application



2. Bend

Bend and conform the blanket on or around the patient to cover the greatest amount of skin surface area.



5. Secure

Use tie strips to secure blanket if needed.



8. Clamp

Use cord tether and clamp to secure hose.



Final Lateral Shoulder Application



3. Remove

Remove adhesive strip backing.



6. Unfold

Unfold the head drape and place over the intubated patient's head and neck.*
*Since the adhesive strip should always face the surgical site,

the head drape is unable to be used in certain configurations.



9. Inflate

Turn on the warming unit to inflate the blanket and begin therapy.



Final Robotics Application