

# Application Steps

3M™ Bair Hugger™ Multi-position Upper Body Warming Blanket - Model 62200



## 1. Place

Once surgical positioning is complete, place the blanket on the patient. The small holes should be in contact with the patient's skin.



## 2. Bend

Bend and conform the blanket on or around the patient to cover the greatest amount of skin surface area.



## 3. Remove

Remove adhesive strip backing.



## 4. Adhere

Adhere to patient in the desired location (adhesive strip toward surgical site).



## 5. Secure

Use tie strips to secure blanket if needed.



## 6. Unfold

Unfold the head drape and place over the intubated patient's head and neck.\*

\*Since the adhesive strip should always face the surgical site, the head drape is unable to be used in certain configurations.



## 7. Connect

Connect hose to blanket using desired port (not over groin area).



## 8. Clamp

Use cord tether and clamp to secure hose.



## 9. Inflate

Turn on the warming unit to inflate the blanket and begin therapy.



Final Beach Chair Application



Final Lateral Shoulder Application



Final Robotics Application