

Pink and powerful.

Clinpro™ Sealant



Summary of advantages

- Colour-change technology to visualise placement
- Convenient direct-delivery syringe for easy application
- Low viscosity to flow easily into pits and fissures
- Contains and releases fluoride
- Bonds to enamel

Protect teeth against decay.


With its patented colour-change technology, 3M™ Clinpro™ Sealant makes sealant application easier than ever. This fluoride-releasing sealant goes on pink and cures to a natural white. Applied with our convenient ultra-fine syringe tip, its low-viscosity formula flows easily into pits and fissures, yet is viscous enough to stay in place during application.

Backed by over 15 years of clinical success, 3M is a market leader in preventive products. We are committed to providing the most advanced and patient-friendly solutions to dental professionals and their patients.

Before



After

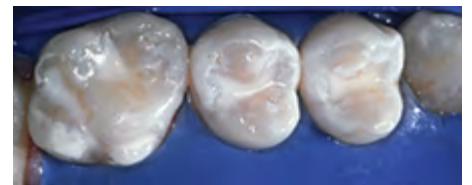

Photography by Dr. Chris Bryant



Easy-to-use technique

Clinpro Sealant offers colour-change technology for ultimate placement control.

Tooth preparation: Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry.


Note: Do not use any cleaning medium that contains oils or fluoride. Prepare adhesive according to dispensing instructions.



1 Apply etchant to all enamel surfaces to be sealed.



2 Thoroughly rinse teeth to remove etchant, then dry the etched surfaces.

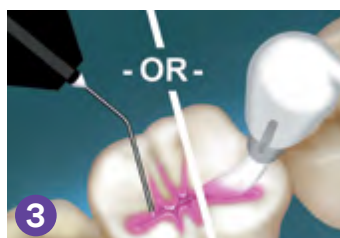

3 Apply Clinpro™ Sealant into pits and fissures.



4 Light cure the sealant with an LED light for the amount of time specified to cure the sealant.

Ordering Information

Item # Product Information

12646 Clinpro™ Sealant Introductory Kit, Syringes

Contains: 2 x 1.2 ml Clinpro Sealant Syringes; 1 x 3 ml 3M™ Scotchbond™ Etchant Syringe; 20 x Sealant Syringe Tips (Black); 25 x Etchant Syringe Tips (Blue)

12647 Clinpro™ Sealant Refill, Syringes

Includes: 1 x 1.2 ml Clinpro Sealant Syringe; 10 x Sealant Syringe Tips (Black)

12642 Clinpro™ Sealant Refill, Bottle

Contains: 1 x 6 ml Clinpro Sealant Bottle

12626T Clinpro™ Sealant - Dispensing Tips

Contains: 10 x Dispensing Tips



3M Oral Care

3M Australia Pty Limited
Building A, 1 Rivett Rd
North Ryde, NSW 2113
Ph: 1300 363 454
3M.com.au/oralcare

3M New Zealand Limited
94 Apollo Drive
Rosedale, Auckland 0632
Ph: 0800 80 81 82
3M.co.nz/oralcare

Always follow the Instructions For Use.
Claims supported by data on file.

3M Science. Applied to Life, 3M, Clinpro,
and Scotchbond are trademarks of 3M.
Please recycle. Printed in Australia.
© 3M 2017. All rights reserved.