



# CuroS™

Disinfecting Port Protectors

## HOW TO USE: DISINFECTING CAP FOR HOME INFUSION

# CuroS Jet™

Disinfecting Cap for Needleless Connectors

**Disinfects in 1 minute — protects ports for up to 7 days.**



### Peel

the foil from the 3M™ CuroS Jet™ Disinfecting Cap



### Push & Twist

the CuroS Jet disinfecting cap clockwise onto the end of the needleless connector.

- If the valve becomes soiled with blood or other contaminants, it should be swabbed with a 70% isopropyl alcohol pad for 15 seconds before placing a new CuroS Jet disinfecting cap on the clean valve.
- As a precaution, replace the CuroS Jet disinfecting cap if it gets wet.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for a minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the CuroS Jet disinfecting cap that has been in place for a minimum of one (1) minute.
- Always place a new CuroS Jet disinfecting cap on needleless connector following completion of a valve access.
- Dispose of the CuroS Jet disinfecting cap after every use.

**WARNING: To avoid potential for injury, use only on needleless connectors.**

**CAUTION: Potential choking hazard.**



## Green means clean.



Critical & Chronic Care Solutions Division  
3M Health Care  
2510 Conway Avenue  
St. Paul, MN 55144 USA

1800 228-3957  
www.3M.com/C3SD

Printed in U.S.A. Please recycle this poster.  
© 3M 2017. All rights reserved.  
3M and CuroS are trademarks of 3M.  
70-2011-6879-9

STERILE	R
---------	---

Active Ingredient: 70% isopropyl alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.