

Efficiency Meets Excellence: Streamlining Posterior Restorations.

Your time matters. So to help keep you on schedule, we simplified the procedure you perform most frequently – the posterior restoration. By using these easy-to-use 3M products together, you'll move through posterior restorations with speed and simplicity. And with Filtek™ One Bulk Fill Restorative at the heart of the process, you'll achieve outstanding results without compromising aesthetics.




3M™ Scotchbond™ Universal Plus Adhesive

The first radiopaque universal adhesive. Versatile. High bond strength. One simple technique and quick 35-second application. Virtually no post-operative sensitivity.



3M™ Filtek™ One Bulk Fill Restorative

Simplicity of bulk placement up to 5 mm deep. Low wear for durability and stress relief. Optimised opacity for greater aesthetics.

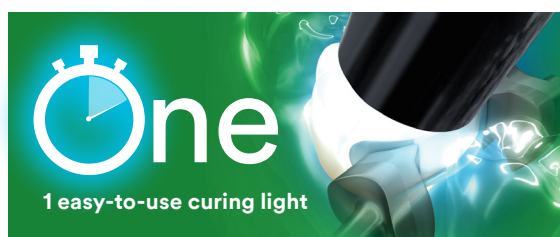

3M™ Elipar™ DeepCure-S LED Curing Light

Predictable, reliable cure. Optimised light guide is more comfortable for patients and provides easier access to posterior teeth.



3M™ Sof-Lex™ Diamond Polishing System

Easily adapts to irregular, convex and concave tooth surfaces and imparts paste-like gloss with the convenience of a rubberised system. Fast and easy to use.



3M™ Clinpro™ Tooth Crème 0.21% Sodium Fluoride Anti-Cavity Toothpaste

Contains 3M's Tri-Calcium Fluoride (TCP). It replaces the toothpaste in the normal brushing routine, making compliance easy. Non-prescription strength.



Ordering Information



3M™ Filtek™ One Bulk Fill Restorative

Refill	Item #	Item #
SHADE	SYRINGE	CAPSULE
A1	4866A1	4867A1
A2	4866A2	4867A2
A3	4866A3	4867A3
B1	4866B1	4867B1
C2	4866C2	4867C2

SYRINGE SHADE REFILLS includes: 1 – 4g syringe; Technique Guide; Instructions for Use.
CAPSULE SHADE REFILLS includes: 20 – 0.2g capsules; Technique Guide; Instructions for Use.



3M™ Scotchbond™ Universal Plus Adhesive – Vial

Item #	Product Information
41293	Intro Kit Vial Includes: 1 Vial 5ml, 50 Disposable Brushes, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41294	Refill Vial – 1 x 5ml
41295	Refill Vial – 3 x 5ml
41263	3M™ Scotchbond™ Universal Etchant Etching Gel Includes: 2 Syringes 3ml, 50 Dispensing Tips

3M™ Scotchbond™ Universal Plus Adhesive – Unit Dose

Item #	Product Information
41297	Intro Kit Unit Dose Includes: 50 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41298	Unit Dose 100 Includes: 100 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41299	Unit Dose 200 Includes: 200 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips
41304	Unit Dose 400 Includes: 400 Unit Doses, 1 Etchant Syringe 3ml, 25 Dispensing Tips



3M™ Elipar™ DeepCure-S LED Curing Light

Item #	Product Information
76978	3M™ Elipar™ DeepCure-S LED Curing Light Includes: Handpiece (Cordless), Charging Base (230V); Li-ion Battery; 10mm Light Guide; Eye Shield
76981	3M™ Elipar™ DeepCure-S Light Guide, 10mm
76984	3M™ Elipar™ DeepCure Eye Shield
76985	3M™ Elipar™ DeepCure-S Rechargeable Li-ion Battery



3M™ Sof-Lex™ Diamond Polishing System

Item #	Product Information
5092-IM	3M™ Sof-Lex™ Diamond Polishing System – Introductory Kit Kit includes: 5 – 3M™ Sof-Lex™ Pre-Polishing Spirals (Beige); 5 – 3M™ Sof-Lex™ Diamond Polishing Spirals (Pink); 1 – 3M™ Sof-Lex™ RA Mandrel; Instructions for Use; Technique Card
5090	3M™ Sof-Lex™ Pre-Polishing Spirals – Refill Includes: 15 Sof-Lex Pre-Polishing Spirals (Beige)
5091	3M™ Sof-Lex™ Diamond Polishing Spirals – Refill Includes: 15 Sof-Lex Diamond Polishing Spirals (Pink)



3M™ Clinpro™ Tooth Crème 0.21% Sodium Fluoride Anti-Cavity Toothpaste

Item #	Product Information
12218	Vanilla Mint Flavor – 113g (4oz) tube



3M™ Clinpro™ 5000 1.1% Sodium Fluoride Anti-Cavity Toothpaste

Item #	Product Information
12214	Vanilla Mint Flavor – 113g (4oz) tube



3M Oral Care

3M Australia Pty Limited
Building A, 1 Rivett Rd
North Ryde, NSW 2113
Ph: 1300 363 454
www.3M.com.au

3M New Zealand Limited
94 Apollo Drive
Rosedale, Auckland 0632
Ph: 0800 80 81 82
www.3M.co.nz

Request
a demo



3M, ESPE, Clinpro, Elipar, Filtek, Scotchbond and Sof-Lex are trademarks of 3M. Used under license.
© 3M 2024. All rights reserved. Information correct at time of printing. Claims supported by 3M data on file. Please refer to the Instructions For Use (IFU). All other trademarks are property of their respective holders.