


The 3M™ Bair Hugger™ Temperature Monitoring System Placement Instructions

Connect the Bair Hugger™ temperature sensor to the sensor cable. Carefully insert sensor tab into the cable. Ensure purple dot is flush with sensor.
 (Caution: do not bend the sensor tab.)

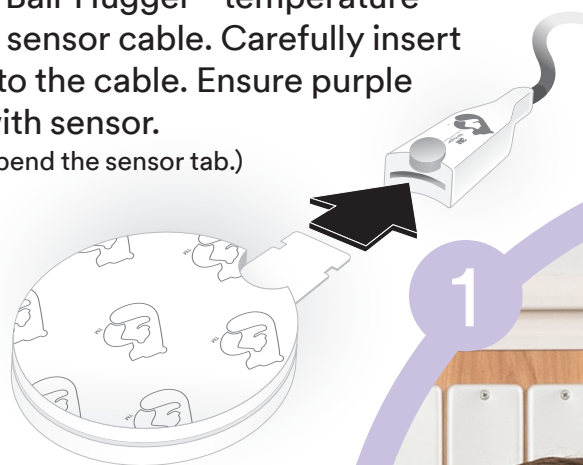


1

Prep sensor site with alcohol wipe, allow to dry.

2

4

Place sensor above orbital ridge, on either side, avoiding frontal sinus.

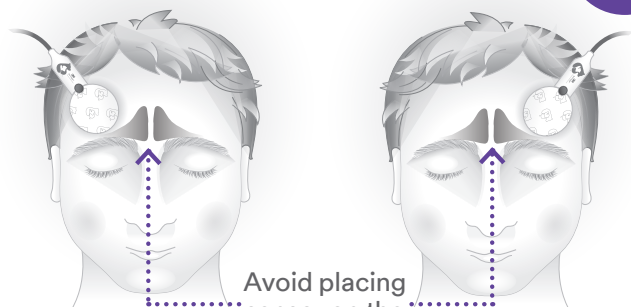

Avoid placing sensor on the frontal sinus, as this may affect sensor accuracy.

3

Remove adhesive liners.



Warnings and Cautions

⚠ Caution: To reduce the risks associated with skin maceration:

- Do not use the sensor on damaged or compromised skin

⚠ Warning: To reduce the risks associated with pressure, and crushed or entangled cords and connectors:

- Do not use a headband or other device to secure the sensor to the patient

⚠ Warnings: To reduce the risks associated with incorrect system or sensor use:

- Limit use of the sensor to 24 hours
 (Extended use may compromise skin, cause degradation of material or performance)
- Avoid exposing the temperature monitoring sensor, sensor cable, power supply and control unit to surgical skin preparation solutions or other fluids
- Do not reposition the sensor
 (Repositioning may weaken the sensor adhesive, damage the sensor or compromise device performance)

This placement guide is intended only as a quick reference. Refer to the product Instructions for Use for complete instructions, and other important information related to the use of the Bair Hugger™ Temperature Monitoring System.

www.BairHugger.com

Available in Canada from:

3M Infection Prevention Division
3M Canada
 300 Tartan Drive
 London, Ontario N5V 4M9
 Canada
 1-800-364-3577

3M Health Care
Infection Prevention Division
 St. Paul, MN 55144-1000
 USA
 1-800-228-3957

3M, Bair Hugger and the Bair Hugger logo are trademarks of 3M. Used under license in Canada.
 © 2017, 3M. All rights reserved. 1711-10377 E