# 7-Step Quick Reference Guide

3M™ Attest™ 390 Auto-reader and 3M™ Attest™ Rapid Readout Biological Indicator (BI)


**ATTENTION:** Wear safety glasses and gloves. **DO NOT** place another label or indicator tape on the vial or on the cap of the BI.

1		<b>Cool</b>	<ul style="list-style-type: none"> <li>▶ Allow BI to cool at least 10 minutes.</li> <li>▶ Crushing or excessive handling of the BI before cooling may cause the glass ampoule to burst.</li> </ul>
2		<b>Close</b>	<ul style="list-style-type: none"> <li>▶ While holding BI in hand, press the cap down.</li> </ul>
3		<b>Crush</b>	<ul style="list-style-type: none"> <li>▶ Hold BI by the cap and use “Crusher Wall” to crush the BI only one time.</li> <li>▶ <b>DO NOT</b> use your fingers to crush the glass ampoule.</li> </ul>
4		<b>Tap</b>	<ul style="list-style-type: none"> <li>▶ Hold the BI by the cap and tap bottom of vial on countertop until the media wets spore strip at bottom of vial.</li> <li>▶ <b>DO NOT</b> roll BI between fingers.</li> </ul>
5		<b>Incubate</b>	<ul style="list-style-type: none"> <li>▶ Place BI into incubation well that corresponds with colour-coded BI cap. Observe LCD panel to confirm BI is correctly seated. Remaining incubation time, in minutes, will be displayed directly below BI on the LCD display to indicate incubation is in progress.</li> <li>▶ <b>DO NOT REMOVE BI</b> from the incubation well until the (+) or (-) symbol on the LCD panel indicates the test is complete.</li> </ul> <p><i>NOTE: Do not reincubate the same BI for a second reading as any subsequent readings would be considered invalid.</i></p>
6		<b>Interpret Results</b>	<ul style="list-style-type: none"> <li>▶ Record result for both test BI (processed) and positive control (unprocessed).</li> </ul>
7		<b>Record</b>	<ul style="list-style-type: none"> <li>▶ Record result for both test BI (processed) and positive control (unprocessed).</li> </ul>

3M™ Attest™ 390 Auto-reader ▶

3M™ Attest™ Rapid Readout Biological Indicator (BI) ▶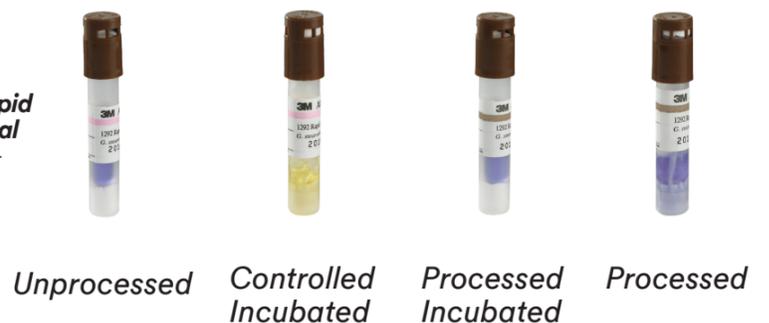

Refer to 3M™ Attest™ Rapid Readout Biological Indicator (BI) package inserts for complete instructions.  
Refer to 3M™ Attest™ 390 Auto-reader Operator’s Manual for complete instructions.

Available in Canada from:  
**3M Infection Prevention Division**  
**3M Canada Company**  
P.O. Box 5757  
London, Ontario N6A 4T1 Canada  
1 800 563-2921

**3M Center**, Building 275-4W-02  
St. Paul, MN 55144-1000 USA  
1 800 228-3957  
3M.com/HealthCare

3M, 3M Science. Applied to Life. and Attest are trademarks of 3M. Used under license in Canada. Please recycle. Printed in Canada.  
© 2017, 3M. All rights reserved. 1708-09713-E

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