

3M Science.
Applied to Life.™

Proven performance ... 15 years and counting

3M™ Clinpro™ Sealant



Did you know ...

ADA and AAPD guidelines now conclude sealants are effective in preventing and arresting non-cavitated carious lesions.

The new industry guidelines support what 3M and dentists like you have been proving for more than 15 years with Clinpro Sealant.

As a market leader in preventive products, 3M Oral Care is committed to providing advanced and patient-friendly solutions like Clinpro Sealant to help you protect the oral health of your patients.

*2016 clinical practice guideline developed by the American Dental Association Council on Scientific Affairs (ADA CDA) and the American Academy of Pediatric Dentistry (AAPD)

“ Sealants are effective in preventing and arresting pit-and-fissure occlusal carious lesions of primary and permanent molars in children and adolescents and adults compared to the non-use of sealants or use of fluoride varnishes ... and, sealants can minimize the progression of non-cavitated occlusal carious lesions (initial lesions) of the tooth that received the sealant.”*

Summary of advantages

- Color-change technology helps you visualize placement
- Flows easily into pits and fissures
- Lower fill rate for fewer occlusal adjustments than filled sealants
- Convenient direct-delivery syringe for easy placement or economic bottle
- Excellent stain resistance
- Contains and releases fluoride
- Can be used with self-etch adhesive or phosphoric acid etch technique
- 10-second cure time with LED lights
- Does not contain BPA

3M's patented color-change technology is just one reason why Clinpro Sealant is an easy choice.

Clinpro Sealant has been proven by numerous clinical studies and trusted by dentists for **over 15 years**. That says a lot about the product performance. It also says that dentists like using it—and for a lot of reasons, like those listed in the summary of advantages.

Goes on pink to guide placement



Cures to white




Clinical photography by Dr. Chris Bryant.

Easy-to-use technique

3M™ Clinpro™ Sealant offers color-change technology for ultimate placement control.

Tooth preparation: Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry.

Note: Do not use any cleaning medium that contains oils or fluoride. Prepare adhesive according to dispensing instructions.

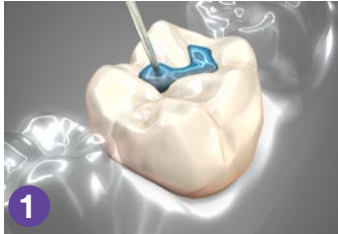

1 Apply etchant to all enamel surfaces to be sealed.


Note: Adhesive may be used instead of etch technique.



2 Thoroughly rinse teeth to remove etchant, then dry the etched surfaces.



3 Apply Clinpro™ Sealant into pits and fissures.

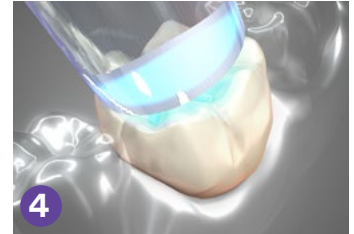

4 Light cure the sealant according to manufacturer's instructions.
LED: 10 sec.
Halogen: 20 sec.

Ordering Information

Item #	Product Information
12626	Clinpro™ Sealant Introductory Kit—Syringes Kit includes: 2 – 1.2ml Clinpro Sealant syringes; 1 – 3ml Scotchbond™ Etchant syringe; 20 sealant syringe tips (Black); 25 etchant syringe tips (Blue)
12627	Clinpro™ Sealant Refill—Syringes Includes: 1 – 1.2ml Clinpro Sealant syringe; 10 sealant syringe tips (Black)
12621	Clinpro™ Sealant Introductory Kit—Bottles Kit includes: 2 – 6ml Clinpro Sealant bottles; 1 – 9ml Scotchbond™ Etchant bottle; 2 brush handles; 60 brush tips; 50 fiber tips; 1 mixing well with cover
12622	Clinpro™ Sealant Refill—Bottle Includes: 1 – 6ml Clinpro Sealant bottle
12626TP	Clinpro™ Sealant and Adper™ Prompt™ Trial Kit Kit includes: 1 – 1.2ml Clinpro Sealant syringe; 10 sealant syringe tips (Black); 20 Adper™ Prompt™ L-Pop™ Applicators*

*MSDS available. Call 1-800-634-2249

www.3M.com/PreventiveCare

3M

3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA
Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada
Phone 1-888-363-3685

3M, ESPE and Clinpro are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Printed in USA. © 3M 2018. All rights reserved. 70-2013-7031-2