

3M Science.
Applied to Life.™

3M™ Coban™ 2 Compression System

Patient information: compression bandaging



**Conformable compression.
Applied to life in motion.**

General advice

Are there any warning signs associated with compression bandaging?

Yes, if you experience any of the following signs and symptoms please contact your healthcare provider straight away, do not delay.

- Pain or excessive tightness
 - Your fingers or toes are blue or white
 - Any tingling or numbness
 - Excessive itching and/or a rash
-

What type of food should I eat?

Eating a balanced diet is important, including consuming starchy food such as bread, pasta and rice. Eating fruit and vegetables helps provide essential vitamins. Meat, fish and eggs provide protein and lastly, dairy products such as milk and cheese are vital to a healthy diet.

Maintaining a healthy weight is good for both conditions as it helps with mobility and reduces stress on joints.



Comfort



Simple application



Increased mobility

Lymphoedema patients

Patients with lymphoedema usually wear compression bandages to help reduce the swelling in their limbs and alleviate the symptoms of heaviness and tightness.

Lymphoedema patients, undergoing an intensive period of bandaging, will differ in treatment time depending on the clinic that they attend. It is normally in the range of between 2–6 weeks.

In lymphoedema, the Coban 2 compression system is changed twice a week, as this has been found to be the most beneficial in terms of reducing swelling.

Exercise is particularly important for patients with lymphoedema. Muscular movement helps move lymph fluid and decreases swelling. Walking, swimming and cycling are all good for overall fitness. Specific exercises for the upper and lower body are available at the following website: www.lymphoedema.org

Yes. After bandaging most lymphoedema patients are fitted for hosiery or a garment that continues to maintain the fluid loss. The benefit of these is that they can be removed in order to bathe.

3M™ Coban™ 2 Compression System

Compression bandaging

Why am I wearing a bandage?

Your healthcare provider has undertaken a full assessment and has prescribed a compression bandage.

How long will I need to wear a bandage for?

How often should the bandaging be changed?

Is exercise important when wearing compression bandaging?

Will I always need to wear some form of compression garment or hosiery?

In both lymphoedema and venous leg ulcers compression must be worn. This will help prevent the risk of swelling, ulcer formation and associated infection from reoccurring in time.

Venous leg ulcers patients

Patients with venous leg ulcers benefit from compression as it reduces swelling, supports the veins, improves blood flow and helps to heal venous leg ulcers.

Venous leg ulcer patients will wear their bandages until the leg ulcer has healed and then for 2–4 weeks post healing. Average healing rates tend to be between 8–12 weeks. However, larger wounds and those that have been present for months or years may take longer to heal.

In venous leg ulcer patients the bandaging is changed weekly. Nurses may elect to bandage more frequently if there is excessive leakage from the wound or presence of infection. There is no evidence to suggest that more frequent bandaging increases healing rates.

Patients with venous ulcers are encouraged to walk in order to activate the calf muscles. This aids blood flow back to the heart and helps heal leg ulcers. Even a little walking around the home and garden can be beneficial. Chair exercises such as flexing the feet up and down at the ankle and rotating the feet can also help. This needs to be done regularly throughout the day.

Yes. Once the ulcer has healed you will be fitted with a compression stocking/sock. This will continue to provide support to your veins and reduce the risk of an ulcer reoccurring.

To learn more about Coban 2 Compression System and to hear what other patients say, please visit **www.3M.co.uk/medical**

Appointment details

Next appointment: _____

Next appointment: _____

Next appointment: _____

Next appointment: _____

Next appointment: _____

Next appointment: _____

Next appointment: _____



3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
Leicestershire
LE11 5RB
01509 611611
www.3M.co.uk/medical

3M Ireland Limited
The Iveagh Building
The Park
Carrickmines
Dublin 18, Ireland
1 800 320 500
www.3M.ie

GF615, J416938.
3M and Coban are trademarks
of the 3M company.
© 3M 2018. All rights reserved.