

3M™ Tegaderm™ Non-Adherent Contact Layer

Details

- Minimises disruption of granulating tissue
- Lessens patient pain during dressing changes and removal
- Maintains an optimal moist wound environment for healing
- Indicated for both partial and full thickness wounds
- Easily cut to size

Non-Adherent Contact Layer

Provides a unique concept in wound management, rapidly transferring wound exudate into an absorbent secondary dressing. Clinical studies show Tegaderm™ Contact Layer can be left on the wound for up to seven days.

Protects the wound bed while reducing pain of dressing changes. A woven, nylon fabric with sealed edges that is non-adherent, non-irritating and hypoallergenic. The wound contact material is placed over the wound and under gauze or other absorbent dressing

Suggested Applications

- Partial and full thickness wounds
- Clean closed surgical incisions
- Superficial and partial-thickness burns
- Donor sites
- Graft fixation sites
- Skin tears
- Traumatic and chronic wounds
- Dermatologic lesions

Chronic Wounds



Vascular ulcers




Pressure ulcers



Cancerous lesions

Donor Sites, Second Degree Burns and Graft Fixation

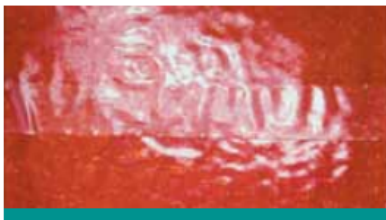

Donor sites



Graft fixation

Surgical Wounds



Incisions

Traumatic Wounds



Abrasions



Application Instructions



Step 1

Position Tegaderm™ Contact Layer over wound.



Step 2

If absorbent filler is needed, fill with 3M™ Tegaderm™ High Integrity Alginate Dressing or other suitable dressing. If moisture is needed for drier wounds, use 3M™ Tegaderm™ Hydrogel Wound Filler.



Step 3

Dress wound with appropriate secondary dressing such as 3M™ Tegaderm™ Foam Dressing (nonadhesive), a 3M™ Tegaderm™ Transparent Film Dressing, gauze or other suitable dressing. Secure secondary dressing as appropriate.



Step 4

Tegaderm™ Contact Layer may be applied under 3M™ Coban™ 2 Layer Compression System or 3M™ Coban™ Self-Adherent Wrap.

Removal Instructions



Step 1

Remove Tegaderm™ Contact Layer by gently lifting. If wound surface appears dry, first moisten material with sterile saline.

Sizes & Codes

| 3M Code | Size | NHS Code |
|---------|----------------|----------|
| 5642 | 7.5 cm x 10 cm | EKH043 |
| 5643 | 7.5 cm x 20 cm | EKH044 |
| 5644 | 20 cm x 25 cm | EKH045 |

