3M

curos™

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2. Maki DG, Kluger DM, Crnich CJ. The risk of bloodstream infection in adults with different intravascular devices: a systematic review of 200 published prospective studies. *Mayo Clin Proc.*

3. Mermel LA. Prevention of Intravascular Catheter-Related Infections. *Ann Intern Med.* 2000; 132:391-402.

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5. Zimlichman, E; Henderson, D et al. Health Care–Associated Infections: A Meta-analysis of Costs and Financial Impact on the US Health Care System. *JAMA Intern Med.* Published online September 02, 2013

6. Kaler, W. Making it easy for nurses to reduce the risk of CLABSI, *Patient Safety, and Quality Healthcare.* 2014; 11 (6), 46-49.

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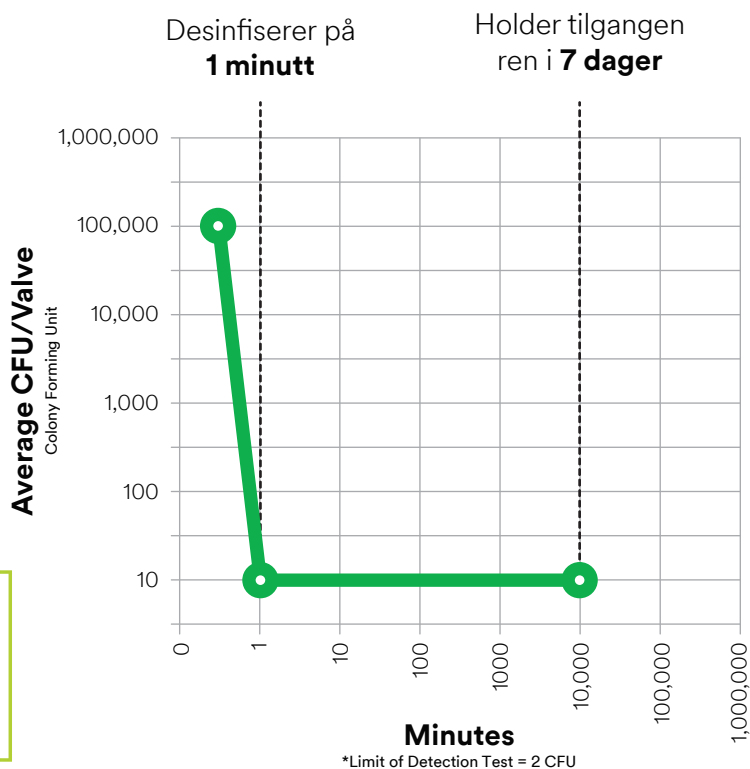
8. Data reflects *in vitro* findings on Curoso™ Disinfecting Port Protectors conducted by an independent laboratory

9. For more information regarding organisms associated with catheter-related bloodstream infections, refer to Wenzel RP and Edmond MB. The Impact of Hospital-Acquired Bloodstream Infections. *Emerg Infect Dis.* 2001 Mar Apr;7(2):174-7.

10. J. Lee, “Disinfecting cap makes critical difference in central line bundle for reducing CLABSI,” in Proceedings of the APIC Annual Conference vol. 39, p. E64. Fort Lauderdale, Fla, USA, 2013.

Alle pasienter, alle tilganger, alltid!

Bruk hele Curoso-familien for å redusere risikoen for intraluminal smitte.

I henhold til 2016 Infusion Nurses Society Standards of Practice, "Use of passive disinfecting caps containing disinfecting agent (IPA) have been shown to reduce intraluminal microbial contamination and reduce rates of CLABSIs.¹¹" (Level II)

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**“Ensure disinfecting
supplies are readily
available at bedside to
facilitate staff compliance
with port disinfection.”¹¹**

11. Gorski L, Hadaway L, Hagle ME, McGoldrick M, Orr M, Doellman D. Infusion Therapy Standards of Practice. Journal of Infusion Nursing. 2016; 39(suppl 1):S1-S159.

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63% to 80%

American Journal of Infection Control: Volume 40 Number 12; December 2014

Impact of Universal Disinfectant Cap Implementation on Central Line-Associated Bloodstream Infections

Katreena Collette Merrill RN, PhD, Sharon Sumner RN, BS, Lorraine Linford RN, BS, CNSC, Carrie Taylor RN, MS, CIC, Christopher Macintosh RN, BS.

- The rate of CLABSI infections decreased by >40% following implementation of the 3M™ Curo™ Disinfecting Strip for Needleless Connectors (IRR = .557, P = .004).
- Curo Cap use was associated with an estimated savings of almost \$300,000 per year in the hospital studied.
- Weekly audits of compliance demonstrated that a 10% increase in nurse compliance resulted in a statistically significant 7% drop in infection rate.

The Journal of the Association for Vascular Access: Volume 17 Number 4; December 2012

Central Venous Catheter Protective Connector Caps Reduce Intraluminal Catheter-Related Infection

*Chuck Ramirez, BA, RRT, VA-BC, Antonina M. Lee, MEd, MPH, RN, CIC, Ken Welch, MD
Banner Estrella Medical Center, Phoenix, AZ*

- During 2010, the CLABSI rate reduced from 1.9 in 2010 to 0.5 during the one-year trial period.
- The implementation of 3M™ Curo™ Disinfecting Strip for Needleless Connectors during month five of the trial increased compliance rates from 63% to 80%.

Use of a Central Catheter Maintenance Bundle in Long-term Care Hospitals

Anthony M. Grigonis, PhD, Amanda M. Dawson, PhD, Mary Burkett, DNP, CNS, Arthur Dylag, MA, MBA, Matthew Sears, BS, Betty Helber, RN, MS, ANE-BC, and Lisa K. Snyder, MN, MPH

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\$3.7 MILLION

- A central catheter maintenance bundle was implemented in 30 LTACHs, and compliance with the bundle was tracked for six months. CLABSI rates were monitored for 14 months before and 14 months after the bundle was implemented.
- In addition to the CDC guidelines, the bundle protocol included education on the protocol, mandatory use of alcohol-based central catheter caps, chlorhexidine gluconate dressings, and formation of a central catheter team of nurses.
- A mean reduction of 4.5 CLABSIs per LTACH occurred for the LTACHs studied for 14 months after the bundle was implemented. This infection reduction could translate to a savings of approximately \$3.7 million annually for the 30 LTACHs studied and could have potentially saved 20 patients' lives, assuming a 15% mortality rate from CLABSIs.

Impact of Alcohol Impregnated Port Protectors and Needleless Neutral Pressure Connectors on Central Line-Associated Bloodstream Infections and Contamination of Blood Cultures in an Inpatient Oncology Unit

Michael A. Sweet, PharmD; Aaron Cumpston, PharmD; Frank Briggs, PharmD; MPH, Michael Craig MD and Mehdi Hamadani, MD

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\$500,000

- A total of 6,851 central line-days and 16 CLABSIs (2.3 infections/1,000 central line days) were documented during the control period, compared with 3,005 central line days and one CLABSI (a rate of 0.3 infections/1,000 central line days) during the intervention period (relative risk, 0.14; 95% confidence interval [CI], 0.02-1.07; P = .03).
- This 32-bed study showed \$500,000 in annualized savings (Sweet MA, et al. SHEA Product Evaluation 2011).
- The rate of contaminated blood cultures from central lines was 2.5% (17 of 692) during the control period, but only 0.2% (1 of 470) during the intervention period (relative risk, 0.09; 95% CI, 0.01-0.65; P = .002).
- The rate of adherence to the intervention was 85.2% (228 of 269 patients with catheter protectors).

Port Protectors in Clinical Practice: an Audit

Corinne Cameron-Watson. Barking Havering and RedBridge NHS Trust

Curos desinfeksjon for kateterkoblinger er estimert å kunne gi en potensiell tidsbesparelse på

**82.4
ARBEIDSDAGER
PR. ÅR**

- The study measured the effect on compliance and incidence of vascular access device (VAD)-related bacteremia following the introduction of a passive disinfection device (Curos) for 6 months.
- As compared to data collected in a benchmark "scrub the hub" audit, data post Curos cap implementation showed VAD-related bacteremia rates reduced by 69% when staff compliance with Curos cap placement onto VADs was 80% or more.
- The use of Curos caps was estimated to provide a potential clinical-time saving of 659.4 hours per year, which equates to 82.4 working days per year (based on an 8-hour day).
- Of the 86 staff trained to use a port protector, 70% returned completed questionnaire, and of these 100% preferred the port protectors to manual scrubbing.

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3M og Curos er varemerker fra 3M. Tego® er et varemerke fra ICU Medical, Inc. 3M™ Curos™ desinfeksjonshetter er ikke tilknyttet ICU Medical, Inc. Printet i Norge. Vennligst resirkuler. © 3M 2017. Alle rettigheter. BP531157-INO170607.