

## Safety Considerations for Using 3M™ Cavilon™ No Sting Barrier Film for General Use

### **Before use, review the product insert\*:**

- 3M™ Cavilon™ No Sting Barrier Film is extremely flammable until it has completely dried on the skin. Allow the product to dry for approximately 30 seconds. If an odor is detectable, the product is not completely dry.
- Do not use in settings where static discharge is anticipated or likely.
- Do not smoke when using this product.
- Do not use near flames-e.g. heaters with a pilot light, candles, etc.
- If used in a setting where there are devices that could be an ignition source or produce heat, e.g. electrocautery:
  - ALLOW THE PRODUCT TO DRY FOR A MINIMUM OF 90 SECONDS *BEFORE* USING ANY DEVICE THAT COULD PROVIDE A SOURCE OF HEAT OR IGNITION. THIS ALLOWS FLUID TO DRY AND VAPORS TO DISSIPATE.

### **Other considerations:**

- Skin should be clean and dry prior to application of Cavilon No Sting Barrier Film.
- If an area is missed, reapply to that area only after the first application of Cavilon No Sting Barrier Film has completely dried. If an odor is detectable, the product is not completely dry.
- If Cavilon No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow the film to thoroughly dry before returning skin to the normal position.

\*A product insert is provided in each box. You may also download an electronic copy of the product insert from [3M.com/IFUFinder](http://3M.com/IFUFinder).

**3M Medical Solutions Division**  
2510 Conway Ave.  
St. Paul, MN 55144 USA

Phone 1-800-228-3957  
Web 3M.com/Cavilon

3M and Cavilon are registered trademarks of 3M.  
Please recycle. Printed in USA © 3M 2018.  
All rights reserved. 70-2011-7172-8

