


# 3M™ Tegaderm™ Foam Dressing Selection Guide

Improper use of medical adhesives may cause red, irritated skin or Medical Adhesive-Related Skin Injury (MARSI), which may cause pain, increase the risk of infection and delay healing. That's why it's important to choose the right dressing for both the patient and clinical need. Use the diagram below to find the right 3M™ Tegaderm™ foam dressing.



## Wounds

Fragile/at-risk skin

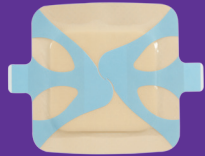

**3M™ Tegaderm™ Silicone Foam Dressings**

**Why they're ideal:**  
 Featuring 3M's unique silicone adhesive, these dressings provide strong yet gentle adhesion that's kind to at-risk skin, helping to reduce the risk of MARSI.

Intact skin



**3M™ Tegaderm™ High Performance Foam Adhesive Dressings**

**Why they're ideal:**  
 Featuring 3M's acrylate adhesive, these dressings provide secure adhesion for longer wear time – even in moist, high-shear locations – meaning fewer, less frequent dressing changes.



## Compression

With adhesion



**3M™ Tegaderm™ Silicone Foam Non-Bordered Dressings**

**Why they're ideal:**  
 With a gentle adhesive that keeps the dressing securely in place, and multiple layers that absorb moisture while providing extra padding, these dressings can easily be cut to a custom shape or size or used under compression.

Without adhesion



**3M™ Tegaderm™ High Performance Foam Non-Adhesive Dressings**

**Why they're ideal:**  
 With multiple layers that absorb moisture while providing extra padding, these dressings can easily be cut to a custom shape or size or used under compression when adhesion (even gentle adhesion) isn't needed.


# When to change a 3M™ Tegaderm™ foam dressing.

It's not as soon as you may think.

Some clinicians inadvertently change dressings too soon, thinking they're saturated when they still have absorption capacity. These unnecessary dressing changes can add up quickly in terms of waste and cost – impacting the clinical and cost-effectiveness of the dressing.

3M™ Tegaderm™ High Performance Foam Adhesive Dressings and 3M™ Tegaderm™ Silicone Foam Dressings feature an innovative, multi-layer design that wicks moisture away from the wound surface, and distributes and locks moisture on the back of the dressing, limiting backward migration and protecting periwound skin. As the dressing absorbs, there is an increase in the evaporation of moisture through the film backing, which adds to the overall fluid handling ability of the dressing for extended wear time.

Even though the top side of the dressing may show exudate, the wound side will still have absorption capacity left, meaning it's not yet time to change this dressing.

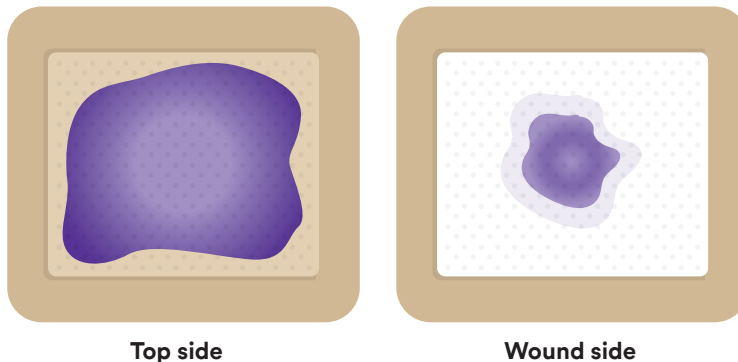

To find out more about 3M™ Tegaderm™ Silicone Foam Dressings please visit [3M.co.uk/SiliconeFoam](https://www.3m.co.uk/SiliconeFoam)



**3M United Kingdom PLC**  
Charnwood Campus  
10 Bakewell Road  
Loughborough  
LE11 5RB  
+44 (0)1509 611 611

3M and Tegaderm are trademarks of the 3M Company.  
© 3M 2018. All rights reserved.

3M™ Tegaderm™ foam dressings are designed with multiple layers that vertically wick and distribute moisture to the back of the dressing. As the dressing absorbs, you'll see discoloration, which is completely normal. When the exudate spreads to the edges of the secondary layer, it's time for a dressing change.

