



FUTURO™ PLANTAR FASCIITIS NIGHT SUPPORT

48507



FIRM
SUPPORT



MOISTURE
WICKING



BREATHABLE

INTENDED USE:

To support the foot, arch and heel

IDEAL FOR:

Ideal for Nighttime support while sleeping

KEY FEATURES:


- 1 Designed to comfortably hold foot at the recommended position.
- 2 Comfortable, adjustable fit.

SIZING:


Category Number	Inches	CM
48507	8.0-15.0	20.3-38.1

Adjust to fit.
Fits left or right foot.
Measure the circumference approximately 5cm above the ankle bone.


HOW TO WEAR:

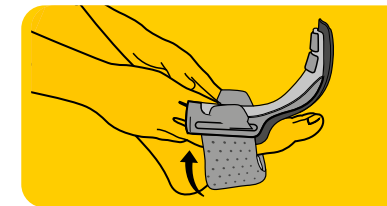

Loosen fasteners on calf and foot straps.
Slide foot through calf strap so support rests on top of foot.

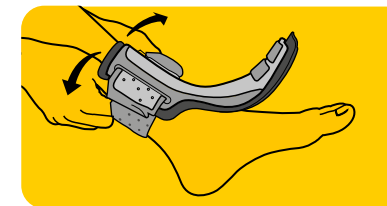

Adjust position of support so calf strap is approximately 5cm (2 inches) above the ankle bone. Adjust calf strap from both sides to provide a firm, yet comfortable fit.

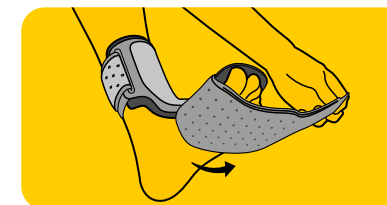

Center wide portion of foot strap under ball of foot. Attach fastener on wider end of foot strap to top of support. Pull toes up, wrap remaining strap around foot and attach fastener.

Contraindications: This product is not recommended for individuals with diabetes, neuropathy or vascular disease. Product not intended for load bearing.
Please consult your physician before using.

Composition: Polypropylene, Polyurethane, Nylon, Polyester, Thermoplastic Elastomer, Spandex.
If discomfort or pain results, persists or increases, discontinue use and consult a physician.
3M and FUTURO are trademarks of 3M.

3M Belgium bvba/sprl
Dép. Consumer Health Care
www.futuro.be

