

Adper™ Scotchbond™ Multi-Purpose Adhesive System



1

Etch:

- Apply Scotchbond™ Etchant to enamel and dentin—wait 15 seconds.
- Rinse.
- Remove excess water with an air syringe or by blotting. Leave moist.



2

Prime:

- Apply Scotchbond™ Multi-Purpose primer to enamel and dentin.
- Dry gently for 5 seconds (*no waiting*). Surface will appear shiny.



3

Bond:

- Apply Adper Scotchbond™ Multi-Purpose Adhesive to enamel and dentin.
- Light-cure for 10 seconds.

Note: Adhesive will bond to Vitrebond™ light cure glass ionomer liner/base whether or not ionomer was treated with etchant and primer.

Adper™ Scotchbond™ Multi-Purpose Adhesive System

3M ESPE Customer Care/MSDS Information:

U.S.A. 1-800-634-2249 and Canada 1-888-363-3685

3M, ESPE, Adper and Scotchbond are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2012. All rights reserved.

3M ESPE

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • www.3MESPE.com



44-0007-7267-1-A