


Adper™ Scotchbond™ Multi-Purpose Plus Adhesive System



Preparation:

- Prepare a standard amalgam preparation. Roughen any existing amalgam and follow the same bonding procedure outlined below for enamel and dentin.
- When bonding amalgam rub the inner surface of the matrix band with wax to prevent inadvertent bonding.
- Place matrix.

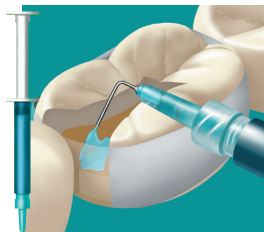

1

Etchant:

- Apply Scotchbond™ Etchant to enamel and dentin—wait 15 seconds.
- Rinse.
- Remove excess water with an air syringe or by blotting. Appearance should be shiny. If surface has matte appearance it should be remoistened.



2

Activator/Primer:

- Mix one drop each of Activator and Primer.
- Apply to etched enamel and dentin — wait 15 seconds.
- Dry gently for 5 seconds.

Adper™ Scotchbond™ Multi-Purpose Plus Adhesive System



3

Adhesive Application:

- Mix one drop each of Adhesive and Catalyst.
- Apply the mixed adhesive to the primed enamel and dentin.
- Thin the mixture using a brush tip, if desired. Avoid excessive air thinning.



4

- Triturate amalgam.



5

Completing the Restoration:

- Condense and burnish the amalgam.

Adper™ Scotchbond™ Multi-Purpose Plus
Adhesive System

Adper™ Scotchbond™ Multi-Purpose Plus
Adhesive System

3M ESPE Customer Care/MSDS Information:**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE, Adper and Scotchbond are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2012. All rights reserved.

3M ESPEDental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • www.3MESPE.com