


Adper™ Scotchbond™ Multi-Purpose and Multi-Purpose Plus Adhesive Systems

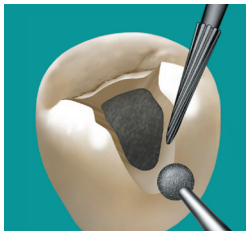

1

Preparation:

- Isolate and clean the tooth.
Use a diamond to remove weakened porcelain. Roughen alloy surfaces. Bevel the margin and remove 1 mm of surface glaze beyond the margin.



2

Etch:

- Apply Scotchbond™ Etchant to porcelain and alloy—wait 15 seconds.
- Rinse.
- Dry.



3

Silane Prime:

- Apply RelyX™ Ceramic Primer to porcelain and alloy surfaces.
- Dry.

Adper™ Scotchbond™ Multi-Purpose and Multi-Purpose Plus Adhesive Systems



4

Bond:

- Apply adhesive to the silane primed surfaces.
- Light-cure for 10 seconds



5

Composite:

- Complete the repair with the Filtek™ restorative of your choice.

3M ESPE Customer Care/MSDS Information:**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE, Adper and Scotchbond are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2012. All rights reserved.

3M ESPEDental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • www.3MESPE.com

44-0007-7266-3-A