

## Gold Anodized Crowns

### Indications:

- Short-term provisional coverage
- Premolars and molars

### Tooth Preparation:

- Prepare tooth and determine mesio-distal width.
- Select appropriate crown and trial fit.

### Crown Preparation:

- Seat crown on preparation. If gingival blanching occurs, trim crown using a 3M™ ESPE™ Crown Scissors. Reseat in mouth and establish occlusion.



- Contour crown using a 3M™ ESPE™ Contouring Pliers.



- Crimp crown margin using a 3M™ ESPE™ Crimping Pliers. Reseat and check fit.



- Outline margin with pencil and trim crown back to this line using scissors.



## Gold Anodized Crowns



- Smooth and polish, as required, using 3M™ ESPE™ Sof-Lex™ Finishing and Polishing Discs, stone, and rubber wheel.



### Crown Cementation:

- Mix cement and fill crown, assuring voids are not present in cement.

**Note:** For custom margins and extended durability, line crown with acrylic or resin material.



- Seat crown and allow the cement to set.



- Remove excess cement with scaler or explorer. Floss interproximal areas.