

3M™ Tegaderm™ Absorbent Clear Acrylic Dressing Application and Removal Guide



Refer to the instructions for use for complete information related to the use of Tegaderm absorbent clear acrylic dressings. Visit 3M.com/IFUFinder.

Knee Application and Removal Instructions

Prior to Application *Note: Follow facility guidelines for infection control.*

1. **Cleanse** the surgical incision and surrounding skin thoroughly. Allow the surrounding skin to dry before applying the dressing.
2. **Evaluate** the surgical incision and select the correct dressing size. The dressing pad should extend past ends of incision by 1 cm (3/8 inch). The acrylic pad is not intended to be cut.



Dressing Application



1 **Open package and remove** sterile dressing. **Peel liner** from the dressing, exposing the adhesive surface.



2 **Position and center** the dressing over the incision, adhesive side down.



3 **Gently press** the dressing in place, smoothing from the center outward.



4 **Slowly remove the frame** while smoothing the film border and applying firm pressure. **Avoid stretching the dressing on the skin.**



5 **Conform and smooth** the dressing, gently pressing down on the edges to ensure optimal adhesion.

TIP

During use, it is normal for fluid to be absorbed throughout the pad. Drainage moving to the edge of the pad **does not require a dressing change**. The dressing should be changed at the discretion of the clinician.

Dressing Removal

Please be advised that adhesive wound closure strips are likely to be removed when the 3M™ Tegaderm™ Absorbent Clear Acrylic Dressing is removed.


1. **Carefully lift the film edges** from the skin. Continue to lift the border of the dressing to remove.
2. Continue lifting film **until all edges are free** from the skin surface.
3. **Remove dressing slowly**, folding over itself.



Abdomen Application and Removal Instructions

Prior to Application *Note: Follow facility guidelines for infection control.*

1. **Cleanse** the surgical incision and surrounding skin thoroughly. Allow the surrounding skin to dry before applying the dressing.
2. **Evaluate** the surgical incision and select the correct dressing size. The dressing pad should extend past ends of incision by 1 cm (3/8 inch). The acrylic pad is not intended to be cut.

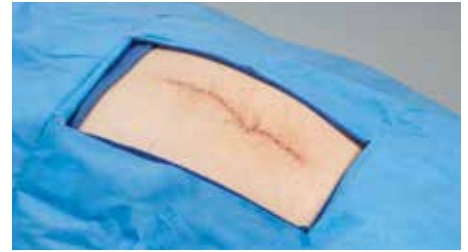

Dressing Application



1 **Open package and remove** sterile dressing. **Peel liner** from the dressing, exposing the adhesive surface.



2 **Position and center** the dressing over the incision, adhesive side down.



3 **Gently press** the dressing in place, smoothing from the center outward.



4 **Slowly remove the frame** while smoothing the film border and applying firm pressure. **Avoid stretching the dressing on the skin.**



5 **Conform and smooth** the dressing, gently pressing down on the edges to ensure optimal adhesion.

TIP

During use, it is normal for fluid to be absorbed throughout the pad. Drainage moving to the edge of the pad **does not require a dressing change**. The dressing should be changed at the discretion of the clinician.

Dressing Removal

Please be advised that adhesive wound closure strips are likely to be removed when the 3M™ Tegaderm™ Absorbent Clear Acrylic Dressing is removed.

1. **Carefully lift the film edges** from the skin. Continue to lift the border of the dressing to remove.
2. Continue lifting film **until all edges are free** from the skin surface.
3. **Remove dressing slowly**, folding over itself.



For more information, contact your 3M Health Care Sales Representative, call the 3M Health Care Customer Helpline at 1-800-228-3957 or visit [3M.com/Medical](https://www.3M.com/Medical).



3M Medical Solutions Division
3M Health Care
2510 Conway Ave.
St. Paul, MN 55144 USA

Phone 1-800-228-3957
Web [3M.com/Medical](https://www.3M.com/Medical)

3M and Tegaderm are registered trademarks of 3M. Please recycle.
Printed in USA. © 3M 2019. All rights reserved. 70-2011-7895-4